

drive

ROOMBA


Indoor Rollator

CE




USER MANUAL

Parts Description

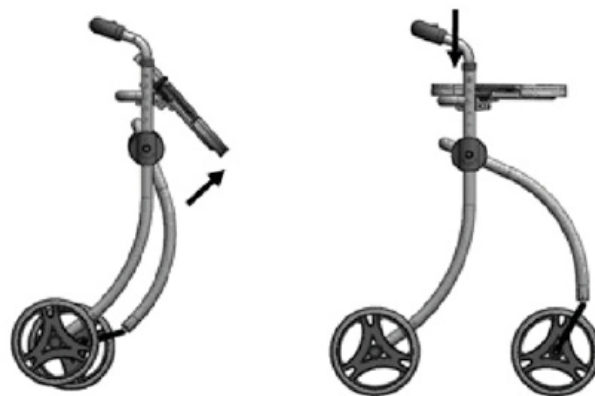

To ensure the rollator is fully open, support the handle with one hand and press on the tray with your other hand. When fully opened, you will hear a 'click' informing you the frame is locked in position.

Folding the Rollator

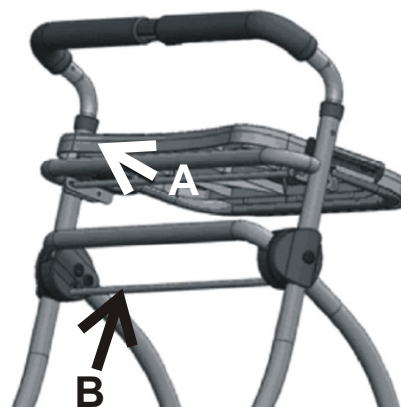

Grasp the tray folding bar and pull in the direction shown by arrow A. The tray will drop down.

To fold the rollator frame, pull the bar B in the direction shown. The rollator frame will then fold up.

Height Adjustment

Handgrip Height Adjustment


Remove the Height Adj Clips from the rollator frame. Reposition the handgrip to the required height and reinsert both of the clips.

Opening and Folding the Rollator

Opening the Rollator

Support the front of the tray with one hand and lift the walker handle upwards, pull the tray in the direction shown in the diagram top right.

Using the Brake

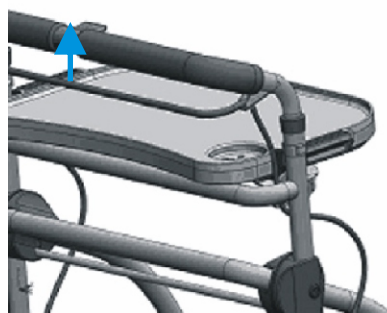


Squeeze the brake lever to apply the brakes (as shown by the blue arrow).

The lever can be operated by both hands or just one hand.

To lock the brake, squeeze the lever to apply the brake and rotate the red Brake Lock so that it holds the lever in position.

Installing the Tray

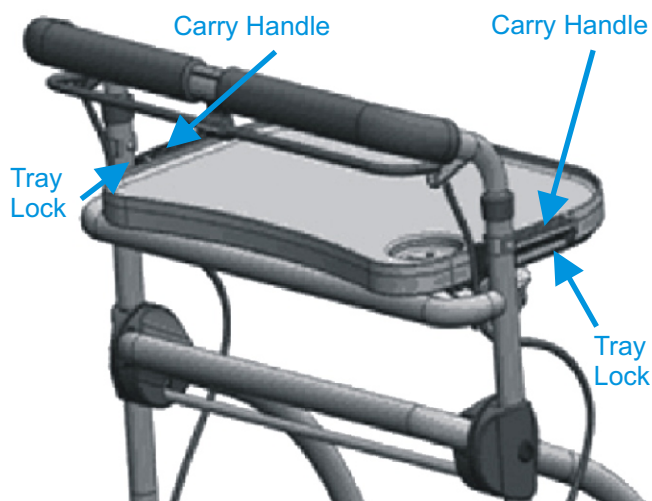

Place the tray on to the frame as shown above. Press down on the two carry handles of the tray to lock.

To remove the tray, press the two tray locks located at each side underneath the carry handles on the tray to release.

The tray can be installed when either the bag is installed or not installed.

Maintenance

- Periodically check that all parts on the rollator are in safe working order.
- Change any damaged or worn parts immediately. Do not use until parts have been replaced.
- Check all screws and fasteners to ensure they are secure and not loose and/or damaged.
- Check all knobs and Height Adj clips are completely inserted in to their holes.
- Clean wheels with water or a mild detergent then dry them with a cloth. If the wheels are noisy or have difficulty in turning, lubricate them with 3-in-1 oil or similar.
- Clean the rollator regularly with a damp cloth and mild detergent. Do not use any aggressive or harsh detergents.
- The product is suitable for reprocessing in line with manufacturer recommendations.

Safety Precautions

- The device should only be used as a walking aid. It should never be used as a wheelchair, nor used to transport people or animals.
- The performance of the brakes can be affected by tyre and cable wear. The brakes should be tested before each use.
- Do not exceed the maximum user weight of 100kg (16 stone).
- The tray should only be used to carry small lightweight items. Hot objects should not be carried.
- The maximum load for the tray and basket is 5kg.
- Do not expose the rollator to temperatures above 40°C for long periods as this can make some parts hot to the touch.

Specification

Dimensions

Overall length inc basket	55cm (21½")
Overall width	57cm (22½")
Width between handgrips	46cm (18")
Handgrip height	81-94cm (32"-37")

Folded Dimensions


83cm (L) x 55cm (W) x 29cm (H)
32½" (L) x 21½" (L) x 11½" (H)

Weight w. Basket and tray 5.6kg (12lb 5oz)

Frame

Aluminium

Product Label

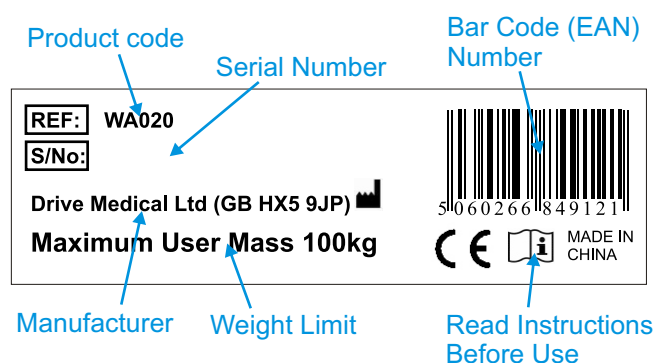

Warranty

Your Drive Medical product is warranted to be free of defects in materials and workmanship for two years from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive Medical stockist.

Manufacturer Details



Drive Medical Ltd,

Ainley's Industrial Estate, Elland,
West Yorkshire HX5 9JP.
www.drivemedical.co.uk

Spare Parts Available



No:	Code:	Description:
1	Z27201-19	Push Handle (Inc. Handgrip, Brake Lever, Red Clip)
2	Z27202-19	Height Adjustment 'E' Clip
3	Z27203-19	Rear Wheel
4	Z27204-19	Front Wheel and Castor Fork
5	Z27205-19	Basket, Black Mesh
6	Z27206-19BLK	Tray, Black
7	Z27207-19R	Tray Lock Bracket, Right
	Z27207-19L	Tray Lock Bracket, Left
8	Z27208-19	Brake Cable
9	Z27209-19R	Brake Unit, Right
	Z27209-19L	Brake Unit, Left