


Product data sheet

Introduction

The Vicair® Strap Back is a flexible back cushion support designed for use with either the Vicair® Liberty back or Vicair® Anatomic back. It can be used on most types of simple to complex wheelchairs. Compatibility can be checked with your technical advisor.

Exploded view

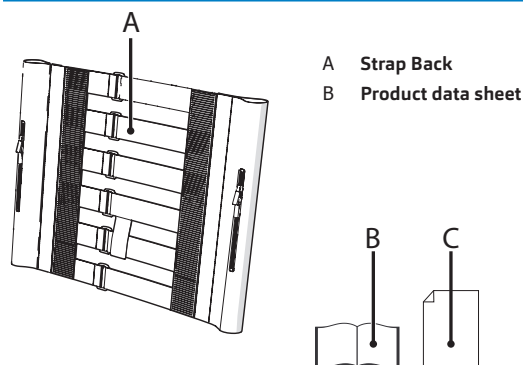

General product specifications

Product name: Vicair® Strap Back
Produced by: Vicair BV, The Netherlands
Warranty: 24 months
Used materials: Fabric and straps made of Nylon. All latex free.



Minimum 15 kg
Maximum 150 kg



Minimum -20°C
Maximum 55°C



50 m (164ft)
below sea level
2400 m (8000ft)
above sea level



Minimum 0%
Maximum 100%

Dimensions in CM		Dimensions in Inches		Strap Back		Used with Vicair® back cushion		Weight
Width	Height	Width	Height	Article number	Article code	Liberty back	Academy Anatomic back	Appr.
30	32	12	12¾	24300132	SB0132	-	AABC/AX/3032	0,2 kg
35 - 40	32	14 - 16	12¾	24300232	SB0232	LBC/1	AABC/AX/3532 - AABC/AX/4032	0,2 kg
45 - 50	32	18 - 20	12¾	24300332	SB0332	LBC/3	AABC/AX/4532 - AABC/AX/5032	0,2 kg
43	32	17¾	12¾	24300432	SB0432	LBC/2	-	0,2 kg
30	40	30	16	24300140	SB0140	-	AABC/AX/3040	0,2 kg
35 - 40	40	35 - 40	16	24300240	SB0240	-	AABC/AX/3540 - AABC/AX/4040	0,2 kg
45 - 50	40	45 - 50	16	24300340	SB0340	-	AABC/AX/4540 - AABC/AX/5040	0,2 kg
30	48	30	19¼	24300148	SB0148	-	AABC/AX/3048	0,2 kg
35 - 40	48	35 - 40	19¼	24300248	SB0248	-	AABC/AX/3548 - AABC/AX/4048	0,2 kg
45 - 50	48	45 - 50	19¼	24300348	SB0348	-	AABC/AX/4548 - AABC/AX/5048	0,2 kg
30	56	30	22½	24300156	SB0156	-	AABC/AX/3056	0,2 kg
35 - 40	56	35 - 40	22½	24300256	SB0256	-	AABC/AX/3556 - AABC/AX/4056	0,2 kg
45 - 50	56	45 - 50	22½	24300356	SB0356	-	AABC/AX/4556 - AABC/AX/5056	0,2 kg

Precautions and safety instructions

en **Attention!** Be sure to read and understand this product data sheet completely.

nl **Let op!** Lees en begrijp dit 'product data sheet' volledig.

de **Achtung!** Bitte lesen Sie das Produktdatenblatt komplett.

se **OBS!** Var noga med att läsa igenom och förstå detta produktdatablad helt.

da **OBS!** Læs dette produktdatablad grundigt igennem.

no ***obs!** Les dette produktblad grundig igennem.

fi **Huomio!** Varmista, että olet lukenut ja ymmärtänyt tuoteselosteen.

fr **Attention!** Assurez-vous d'avoir lu et compris cette fiche dans sa totalité.

es **¡Atención!** Asegúrese de leer y entender esta ficha de producto completamente.


pl **UWAGA !** Pamiętaj , aby w pełni przeczytać i zrozumieć kartę produktu.

Check availability of this product data sheet in your language at: vicair.com/user-manuals

Installation

Attention! Vicair recommends to install and adjust the Vicair® Strap Back under direct supervision of a therapist or technical advisor.

- Ensure the size of the strap back is appropriate for the wheelchair back and the Vicair® back cushion. **1**
- Remove any existing upholstery on the wheelchair back. Fold around the canes ensuring that the Velcro fixation is closed along the full length. The metal eyelets should be at the top of the backrest canes. **2a**
- **Notice!** If a bar is obstructing the installation, use the zippers. **2b**
- If more than one fixation height (i.e. screw holes) is available, ensure the Vicair® Strap Back is at the correct height for the client. **3**
- Fix Strap Back to frame with two screws. If additional screw fixations are required, follow the wheelchair manufacturers recommendations. **4**
- On both sides: lock the two zippers with a tie wrap. **5**
- Apply either the Vicair® Liberty back cushion or Vicair® Anatomic back cushion to the Vicair® Strap Back (follow accompanying product data sheet), and adjust to the client's positioning and comfort requirements.

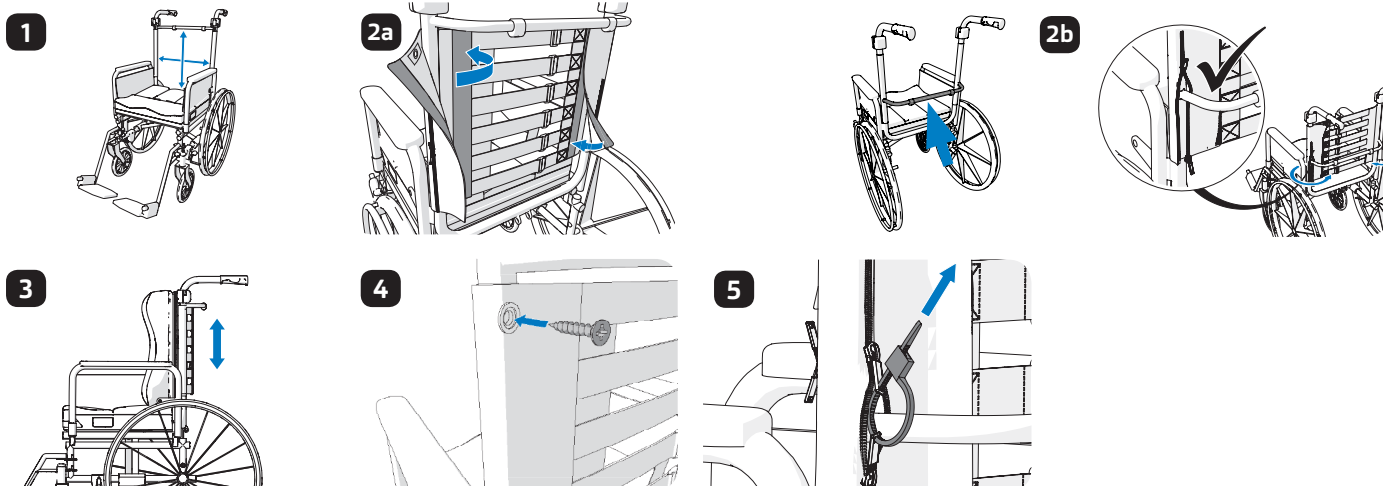

Your feedback is appreciated

Please use the QR code to place your comments about the Vicair Strap Back or its manual