

# nitro rollator

Item#: RTL10266



[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

**drive<sup>TM</sup>**

# Parts List



- |                                   |                        |
|-----------------------------------|------------------------|
| 1. Handgrip                       | 7. Seat                |
| 2. Adjustable tubing handgrip     | 8. Bag                 |
| 3. Rear reflector on handgrip     | 9. Front wheel         |
| 4. Brake lever                    | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel         |
| 6. Back Rest                      | 12. Safety bow         |

# Introduction

The Walking aid is suitable for a single user with limited mobility. The Nitro is ideal for indoor and outdoor use.

## Nitro Includes:

1. Bag
2. Back Rest
3. Operating Instructions

## Unfolding the Nitro

Position the Nitro on its wheels. Open the red safety bow and lift up (figure 1). Press both pipes from the seat downwards until they interlock (figure 2).



## Height Adjustment

Press the red release buttons for handgrips below, as shown in figure 3, and pull the handgrip upwards at the same time. As the handle moves, release the button. This automatically locks in the next possible height. Repeat this process until the desired height is reached.



## Attaching the Carry Bag

Fold the Nitro and push the sliding block from bag into guiding sleeve on the Nitro. Unfold the Nitro. (see figures 4-6).



## Braking/Locking

Pull both brake levers upwards to apply the brakes while the Nitro is moving (see figure 7). Always brake slowly and steady.

Press both brake levers down to apply the locks while the Nitro is stationary (see figure 8). Pull up the brake levers and release to release the locks.

The effect of the brakes can be affected by abrasion of the wheels.

**Regularly test the brakes.**



## Adjustment of Brakes

To adjust the braking strength, adjust the brake by pressing the screw housing downwards (figure 9) and then turn left or right (figure 10).




## Sitting

Position the Nitro on an even and solid surface. Lock the brakes (parking position). The Nitro must stand and must not slip. Sit straight up and do not lean backwards. (See figure 11).



## Folding/Transport/Storage

To fold, pull seat upwards by the handle on the seat plate as shown in figure 12. The Nitro can now be folded complete and secured by using the red safety clasp as shown in figures 14 and 15.

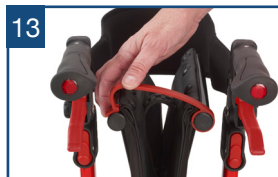

# Safety Instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 300 lbs.
- Regularly check the screws and fasteners on the Nitro and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
- Do not expose the product to temperatures above 104°F for an extended period of time, as this can cause the Nitro parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tires when significantly worn or damaged.
- Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

# Cleaning / Maintenance

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

## specification

Overall Length:	70.5cm (27.75")	Seat Depth:	20cm (8")
Overall Width:	58.5cm (23")	Seat Width:	46cm (18")
Handle (Highest):	97cm (38.25")	Seat Height:	52cm (20.5")
Handle (Lowest):	85cm (33.5")		
		Wheel Diameter:	
Product Weight:	8kg (17.5lb)	Rear:	20cm (8")
Weight Limit:	135kg (21 stone)	Front:	25cm (10")

## parts list

For the full parts diagram, please see [www.drivemedical.co.uk](http://www.drivemedical.co.uk)

Drive UK Code	Description
Z27101-01RD	RTL10266 Brake Handle Ass'y, 12cm length, Red
Z27102-01	RTL10266 Hand Grip, Left, 11.5cm x 5cm
Z27103-01	RTL10266 Hand Grip, Right, 11.5cm x 5cm
Z27104-01	RTL10266 Back Rest Ass'y, Black
Z27105-01RD	RTL10266 Back Rest Knob and Bolt, Red
Z27107-01RD	RTL10266 Back Height Adj Latch, Red
Z27108-01	RTL10266 Seat Canvas Ass'y, Black, 45cm x 27cm
Z27109-01RD	RTL10266 Seat Lifting Strap, Red
Z27110-01RD	RTL10266 Tote Bag, Black with Red Trim, 40W x 20D x 23H
Z27111-01	RTL10266 Tote Bag Mount Set with Screws, 1 pair
Z27112-01	RTL10266 Rear Wheel Ass'y, Black tyre, 19cm x 3.3cm
Z27115-01	RTL10266 Wheel Brake, Left
Z27116-01	RTL10266 Wheel Brake, Right
Z27124-01	RTL10266 Right Fork, Castor & Black Tyre, 25cm x 3.3cm
Z27125-01	RTL10266 Left Fork, Castor & Black Tyre, 25cm x 3.3cm
Z27126-01RD	RTL10266 Locking Latch
Z27127-01	RTL10266 Cross Brace Ass'y, Black
Z27128-01	RTL10266 Cross Brace End Caps With Reflectors, Front, Pair
Z27129-01	RTL10266 Cross Brace End Caps With Reflectors, Rear, Pair
Z27130-01	RTL10266 Handle Bar, Black

## warranty

Your Drive Medical product is warranted to be free of defects in materials and workmanship for two years from date of purchase.

This device was built to exacting stands and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive Medical stockist.



Document code: RTL10266\_IFU (Iss 02/13)

rehab**hire** & **sales**

320 Lorimer Street Port Melbourne Victoria 3207

t: 03 9646 7200 | e: [contact@rehabhire.com.au](mailto:contact@rehabhire.com.au) | [www.rehabhire.com.au](http://www.rehabhire.com.au)