



LITE WHEELCHAIRS

USER MANUAL



ASPIRE LITE

ASPIRE LITE TRANSIT



TABLE OF CONTENTS

1. Key Specifications _____	3
2. Components & Features _____	4
3. Assembly & Use _____	6
4. Troubleshooting _____	7
5. Care & Maintenance _____	8
6. Warnings _____	7
7. Warranty _____	9
8. Compliance _____	9
9. Service Maintenance & Customisation ____	10

Thank you for choosing this Aspire LITE wheelchair.

This product has been designed to offer the maximum in comfort, safety and portability. Aspire LITE and Transit wheelchairs are suitable for indoor and outdoor use. They are easy to fold and extra light for putting in the boot of a car.

It is important that you read this user manual carefully before using your new wheelchair.

1. KEY SPECIFICATIONS

Aspire LITE

Item Code	MWS442300	
Max. User Weight	120kg	
Weight	9.5kg (chair weight)	8kg (transport weight)
Seat Height - adjustable (floor to top of fabric)	FRONT: 460mm	REAR: 435mm
Seat Width	458mm (18")	
Seat Depth	400mm (16")	
Backrest Height (top of seat to top of backrest handle)	465mm	
Overall Width	550mm	
Armrest Height (top of seat to top of armrest pad)	234mm	
Approvals	TGA# 229423	AS/NZS tested and approved
Colour	Blue frame with black upholstery	
Rear Wheel	8" black	

Aspire LITE Transit

Item Code	MWS442301	
Max. User Weight	120kg	
Weight	11.6kg (chair weight)	10.1kg (transport weight)
Seat Height - adjustable (floor to top of fabric)	FRONT: 460mm	REAR: 445mm
Seat Width	458mm (18")	
Seat Depth	400mm (16")	
Backrest Height (top of seat to top of backrest handle)	455mm	
Overall Width	600mm	
Armrest Height (top of seat to top of armrest pad)	234mm	
Approvals	TGA# 229423	AS/NZS tested and approved
Colour	Green frame with black upholstery	
Rear Wheel	12.5" black	

2. COMPONENTS & FEATURES

Aspire LITE



1. 8" Rear Wheels
2. Adjustable Armrests
3. Swing Away Legrests (Removable)
4. Castor Fork
5. 6" Front Castors
6. Folding Foot Plates
7. Attendant Hand Grips
8. Hand Brakes
9. Cross Bar
10. Folding Backrest & Upholstery
11. Seat Upholstery and Lap Belt
12. Carry Pouch on Seat Back

Aspire LITE Transit



1. 12.5" Rear Wheels
2. Adjustable Armrests
3. Swing Away Legrests (Removable)
4. Castor Fork
5. 6" Front Castors
6. Folding Foot Plates
7. Attendant Hand Grips
8. Attendant Hand Brakes
9. Cross Bar
10. Folding Backrest & Upholstery
11. Seat Upholstery and Lap Belt
12. Carry Pouch on Seat Back

3. ASSEMBLY & USE

Open the carton carefully. DO NOT USE a box cutter or sharp implement as this may damage the product. For OH&S, lay the carton on its side with the top of the carton open and slide the plastic bag out containing the wheelchair.

1. Stand the wheelchair in an upright position on the ground



2. Push down on the side seat rails of the wheelchair until the seat is flat to unfold the frame



3. Insert the footrests into the top of the frame receivers at a 90° angle to the chair and swing towards the centre of the chair until the release button engages with a “click” sound



4. TROUBLESHOOTING

Wheelchair is steering/pulling to one side

- Make sure the end user is able to evenly distribute his/her weight on the wheelchair

The wheelchair moves slowly

- Remove any hair or dirt from castor and rear wheel axles
- Make sure there is not too much weight applied to the castors, otherwise adjust the centre of gravity
- Ensure user clothing is not interfering with movement of the rear wheels
- Ensure both brakes are disengaged

The wheelchair does not turn smoothly

- Make sure the front fork bolts are not over tightened which can restrict front castor wheel rotation
- Remove any hair or dirt from castor axles
- Ensure both brakes are disengaged

The brakes do not work properly

- Check distance between tyres and brake foot is sufficient such that the requisite amount of force can be applied to the tyre surface to stop it moving when the brake is in the engaged position

The wheelchair cannot be opened or folded easily

- Clean the crossbar under the seat and apply some grease
- Check that nothing has become lodged under the seat or in and around the cross bar mechanism.
- Check foot plates are folded up

Correct adjustment of the wheelchair is essential for the successful and safe use of this product.

Should assistance be required, contact the Aidacare Service Department on 1300 133 120.

5. CARE & MAINTENANCE

MAINTENANCE SCHEDULE

Weekly	Periodically	Yearly	Serviceable Components
• Check brakes are functioning properly	• Check for signs of wear on front castors • Check for signs of wear on rear tyres	Service to be completed by authorised Aspire service agent	• Wheels • Forks • Axles • Armrest receivers • Seat sling • Footrest mounts • Brakes

CARE FOR YOUR CHAIR

- The nylon seat and backrest upholstery can be removed by simply loosening the screws and washed with a wet sponge and a mild detergent. All other components can be cleaned with a damp cloth.
- Always keep rear wheel axles free from dirt and hair buildup.
- Brakes should be checked regularly.
- If you have any doubts about the correct functioning of the wheelchair or of any its components, please contact AIDACARE on 1300 133 120 or your local Aspire distributor.

6. WARNINGS

MAXIMUM WEIGHT CAPACITY: 120kg - NEVER EXCEED THIS WEIGHT LIMIT

- The foot plates are not designed to be a weight bearing surface. Never stand on the footplates as the wheelchair may tip over.
- Pay attention to your fingers when folding or opening the wheelchair.
- Do not use the brakes to stop a moving wheelchair.
- Never use the wheelchair on escalators, even if accompanied by an attendant.

7. WARRANTY

Aspire LITE wheelchairs have a one year warranty. Please keep the purchase invoice as proof of warranty commencement.

Aidacare is bound by Australian Consumer Law in respect to 'Warranty against Defects' on this product. A copy of the Aidacare 'Warranty against Defects' statement is available off the Home page of our website at www.aidacare.com.au Any contact in regard to warranty matters should be made to Aidacare Customer Service staff on 1300 133 120 or where support will be provided.

Aidacare does not warrant against excessive or incorrect use, modification or any situation that could be deemed as fair wear and tear. Tyres and castor wheels carry a 12 month warranty against manufacturer defects only, NOT wear and tear from reasonable use.

The warranty on this product is a back to base warranty and does not cover any freight costs pertaining to the return of any items under warranty. Aidacare acknowledges and conforms to all ACCC guidelines around consumer rights. For further information contact Aidacare 1300 133 120.

Aidacare will not guarantee the safety and the correct functioning of the wheelchair in the case that any of the original components have been changed or modified and or replaced with non-original Aidacare approved parts. If any faults are detected upon receiving the wheelchair please phone Aidacare to arrange a replacement. Any faults that are detected during normal use should be reported to Aidacare immediately to determine if warranty conditions apply and organise repair and/or replacement as applicable.

8. COMPLIANCE

Aspire Lite wheelchairs are manufactured in ISO compliant production facilities to strict quality control standards.

As part of Australian Standards compliance, safety stickers are located on the frame of this wheelchair. Please ensure that you familiarize yourself with these prior to use.

This product complies with Australian Standard AS/NZS 3695.1:2011. As the importer of this product, Aidacare maintains records of all testing reports and compliance certificates. All testing was completed by Novitatech Children's Services in South Australia.

Novitatech is an independent NATA accredited laboratory.

9. SERVICE MAINTENANCE & CUSTOMISATION

FOR ALL ENQUIRIES, PLEASE CALL 1300 133 120

When sending the chair back for warranty, service or repair work please include all components such as leg rests. Please ensure wheelchair is clearly marked with owner's name and contact details.

To organise pickup by Aidacare, please call 1300 133 120, standard pickup and delivery charges apply.

SPARE PARTS

Aidacare stocks replacement parts for the Aspire wheelchair range. All replacement of parts should be completed by Aidacare or Aspire service technician. The term parts includes but is not limited to: tyres, tubes, brakes, armrests, footrests and upholstery.

A complete parts list is available upon request.

CUSTOMISATION AND MODIFICATION

All wheelchair customisation, modifications and or alterations made to this product are considered as "custom" changes and will void the warranty term. These may cause the product to no longer satisfy Australian Standards and should be considered prior to commencement.

For customisation or modification work, contact AIDACARE on 1300 133 120. Our team of qualified Rehab Engineers are available to assist if required.



AIDACARE PTY LTD
4 Noonan Road, Ingleburn NSW 2565
1300 133 120
www.aidacare.com.au

MADE IN CHINA