These user instructions are available in large print, please contact Homecraft Customer Services for a copy.

Please read these instructions carefully before using this product.

Product Description

The Universal Kettle Tipper has been designed to suit the majority of kettles, with a maximum base diameter of 200mm. However, because kettles are available in such a wide variety of shapes and sizes there may be some instances where a kettle will not fit the product. If you cannot safely fit your kettle to the product it must not be used.

The Universal Kettle Tipper is designed to offer safe assistance to the process of pouring boiled water from a kettle.

Fitting the kettle to the product

This product can be used with jug kettles, cordless kettles or traditional style kettles. If you are using a cordless kettle, it should be left on its base and strapped on as a complete unit. It is recommended that the kettle be emptied prior to fitting to the Universal Kettle Tipper.

1. To adjust the universal kettle tipper, loosen the two thumb screws underneath the Front Clamp so it can be lifted and then slide it forwards.
2. Place your kettle or the kettle base on the Base Plate of the Universal Kettle Tipper and position it centrally. Position any base main lead so that it protrudes from the back of the product and between the Strap Location Hooks.
3. With the kettle positioned centrally on the Base Plate, lift and slide the Front Clamp so that it touches the kettle. Locate the Front Clamp in the serrated slots and tighten the thumb screws. Push the kettle against the Front Clamp if necessary.
4. The kettle is secured to the Universal Kettle Tipper using the straps provided. There is no fixed method to do this but we suggest one strap hooking onto the Front Clamp and going around the girth of the kettle. Then one or both remaining straps through the handle of the kettle and secured between two of the five hook positions on the Base Plate. The straps are all adjustable to ensure they hold the kettle tightly.
5. With the kettle secured to the Universal Kettle Tipper fill the kettle using a separate jug. It is recommended that you only boil the water you need but at least ensure the kettle element is covered by the water.

Please retain these instructions for future reference.

Distributed by: Patterson Medical Ltd, Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK.


UK: Tel: 08448 730 035 Fax: 08448 730 100 www.homecraft-rolyan.com

International: Tel: +44 1623 448 706 Fax: +44 1623 448 784

Manufactured by: Cooper International for Patterson Medical Ltd.

Operation of the Universal Kettle Tipper

1. It is recommended that you practice with cold water first so you are aware of the best way for you to use the Universal Kettle Tipper and to practice the pouring technique. You will also learn where to position the cup for various levels of water and any limitations to positioning the product with regards to the length of your kettle's mains cable.
2. The Universal Kettle Tipper can be operated in two ways. You can choose to hold the kettle by the kettle's own handle and pour it in the normal fashion using the Universal Kettle Tipper for support. Alternatively a side Operating Handle has been provided to give greater leverage. The Operating Handle can be fitted as shown into the pivot socket on either side of the Universal Kettle Tipper. It is also possible to fit the Operating Handle at different 90 degree intervals so that the Universal Kettle Tipper can be operated by pushing forwards, backwards, upwards or downwards on the Handle. Please experiment with the Handle's position to see what works best for you.
3. You are now ready to boil the kettle and use the Universal Kettle Tipper.

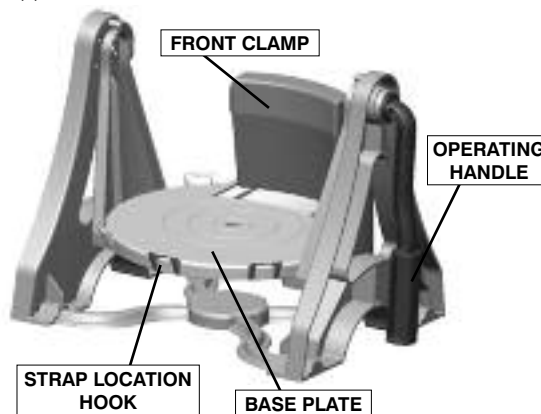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Distribuito da: Patterson Medical Ltd, Nunn Brook Road, Huthwaite, Sutton-in-Ashfield, Nottinghamshire, NG17 2HU, UK
Prodotto da: Cooper International for Patterson Medical Ltd.
Verdeeld door: Patterson Medical Ltd, Nunn Brook Road,
Vervaardigd door: Cooper International for Patterson Medical Ltd.