
BodyAdjust Sleep System by MULTIWAY




Assembly Guide

IMPORTANT NOTE:


Two people are required to set-up and assemble your adjustable, electric bed.

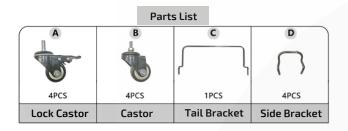

2. Cut off and discard the 4 x zip-tie lines as pictured below (these have secured your bed frame during shipping). Cut the 2 x zip-tie lines from the Tail brackets (ie., mattress retainer brackets – foot end)

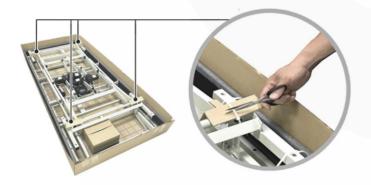

3. Locate the castor box and open. Fit and secure the lock(ing) castors (A) to the frame as indicated in the picture below. Fit and secure the lock(ing) castors (B) to the frame as indicated in the picture below.

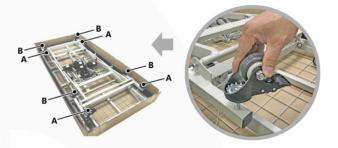


4. Flip the entire bed over [2 x people are required for this procedure]

Lift the entire adjustable electric bed out of the carton, and turn, or flip it over. The bed should end up in a position with the castors on the ground supporting the bed frame, and the frame facing upwards.


Please note: Be careful not to support the bed on one side, or on individual castors during this movement. This will ensure the castors are not damaged or crushed during the manoeuvre by overstressing their bearing load.

6. Insert the Side brackets x 4 (ie., mattress retainers), and the Tail bracket x 1 (ie., mattress retainer - foot end) into position (refer picture below)

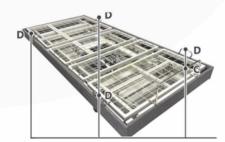

5. Cut off and discard the zip-tie lines securing the power supply, handset control, and moving parts of the frame. The locations of these are found in items 1-6 below.



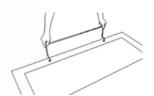



:	lmage no.	Function
:	1	Handset power indicator light. Green = power on/functioning
:	2	Backrest activation: raise
:	3	Backrest activation: lower
:	4	Knee break activation: raise
:	5	Knee break activation: lower
:	6	Trendelenburg functions: tilt with the head raised
:	7	Trendelenburg functions: tilt with the head lowered
:	8	Trendelenburg functions: tilt with foot end raised
:	9	Trendelenburg functions: tilt with the head end lowered
	10	Hi-lo function: raise
:	11	Hi-lo function: lower

Actuators and motors

- Maximum Load: push 6000N
- Warranty: 5 year Frame, 5 year Electrical (Please refer to your MULTiWAY warranty statement)
- Each actuator is serialised. Please keep a central record of these locally for any future service or maintenance enquiries.







General Safety Guidelines & Reminders For Use

Your new electric, adjustable bed base has been quality tested and inspected prior to dispatch. It is highly recommended that these instructions are read thoroughly before you assemble or use your bed, to ensure your piece of equipment performs as per the manufacturer and supplier's intended use, and in accordance with the product features. Adjustable beds are designed to increase user independence and comfort. The type of bed required depends on the specific needs of the use. You are strongly advised to seek guidance from an Allied Health Professional to determine which type of electrical adjustable bed is right for your needs if you have a clinical condition or physical disability.


Please be aware of the following safety precautions to decrease the risk of electric shock, burns, or injury to users, children or nots:-

- 1. Always be aware of your environment. No small children or pets should be present when unpacking or assembling your product.

 All packing and shipping materials need to be disposed of properly after assembly of your product.
- Despite safety mechanisms being in place to reduce the risk of pinching, fingers, hands, arms and legs should not be put underneath the raised head or leg sections at any time.
- 3. Supervision is required when operating your adjustable bed base if children or pets are in the same room. DO NOT OPERATE the bed if people or animals are in the immediate vicinity of the bed base, or underneath the bed base.
- 4. The product has been designed for in-home/facility use. Do not use your electric, adjustable bed base outdoors. This will immediately void any warranty.
- 5. Before leaving the User in the bed, the functions of the bed should be checked to ensure full functionality is available.
- 6. When positioning the bed, ensure that the bed-head and foot sections will not come in contact with fixed objects, such as window sills and shelves, as this can cause structural damage to the bed frames.
- 7. Once the bed is in the correct position, the wheels on the bed should be locked to avoid it rolling away when the user is entering or exiting the bed.
- 8. Unlock the castors before moving the bed base/travelling within the room. Failure to do so will cause the castors to collapse, and all warranties will be voided.
- 9. Electric, adjustable bed bases are simple to use with all functions available on the handset controller.
- 10. Before use, please ensure that an Allied Health Professional has approved the use of each function as users with certain medical conditions or needs may not be suitable for using tilt and leg raise functions.
- 11. Always ensure that the power to the product is off, and that the power cord is detached from any power supply before assembling, using, cleaning, or performing and maintenance on your product.
- 12. Before you disconnect the power cable from the power supply, please return your adjustable bed base to the flat, supine position, and stop operating/moving the position of the bed via the handset controller.
- 13. Because electronics are used on the bed, special care should be taken to avoid getting water on the electrical components.
- 14. Keep the power cord away from water and heated areas. If your adjustable bed base is wet, or the power cord has any fraying, or stress marks, immediately cease using the product and contact your distributer for advice.
- 15. Do not operate your adjustable bed base near explosive gases, or when aerosol spray products are being used.
- 16.Unplug your electronic gadgets and accessories from the USB port (option) of your electric, adjustable base if they are fully charged.
- 17. In the case of a warranty call, MULTiWAY reserve the right to remove and replace the parts with the same, or equivalent components. They may or may not be identical to those previously issued with the product.



BA100 BASS Adjustable Base Only - Long Single 915mm x 2030mm BA101 BASS Adjustable Base Only - King Single 1070mm x 2030mm BA102 BASS Adjustable Base Only - Long Double 1370mm x 2030mm BA103 BASS Adjustable Base Only - Queen 1525mm x 2030mm

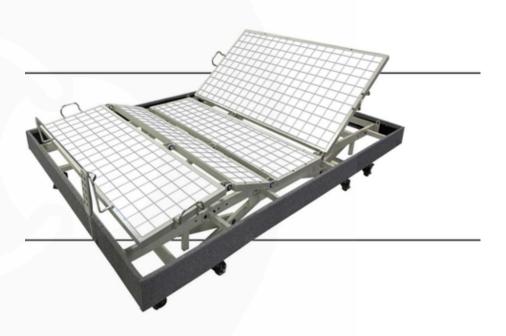

ASSEMBLY AND USER MANUAL

MULTIWAY Trading (AUST) Pty Ltd | P 1300 168 584 | A Unit 1 & 2/12 Commercial Drive, Ashmore QLD 4214 | E sales@multiway.com.au