

rehabhire & sales

YOUR  
LEADING  
HEALTHCARE  
EQUIPMENT  
PARTNER

# Joerns Oxford® Up

Safe and efficient standing,  
transfers and rehabilitation



1300 000 030 | [rehabhire.com.au](http://rehabhire.com.au)

YOUR  
LEADING  
HEALTHCARE  
EQUIPMENT  
PARTNER

## Oxford® Up


### Simple and Safe

The Oxford Up is a uniquely designed manual stand-aid and transfer device, suitable for those who require assistance when standing but are able to contribute effort to the process. By encouraging user participation, the Up can play a supportive role in patient rehabilitation, function, and independence.

### Features and Benefits

- Disassembles for easy storage and transportation
- Adjustable legs for access around furniture
- Contoured swing-away seat pads
- Ergonomic design considerations
- Compatible sling offer for additional support

- A** Multi-point Caregiver Push Handle
- B** Patient Hand-hold Bar
- C** Attachment Point for Compatible Clip Slings
- D** Contoured Swing-away Seat Pads
- E** Soft Moulded Knee Pad
- F** Leg Adjustment Foot Pedal
- G** Braked Rear Castors (100mm)
- H** Base / Footplate with Non-slip Overlay
- I** Swivel Front Castors (75mm)

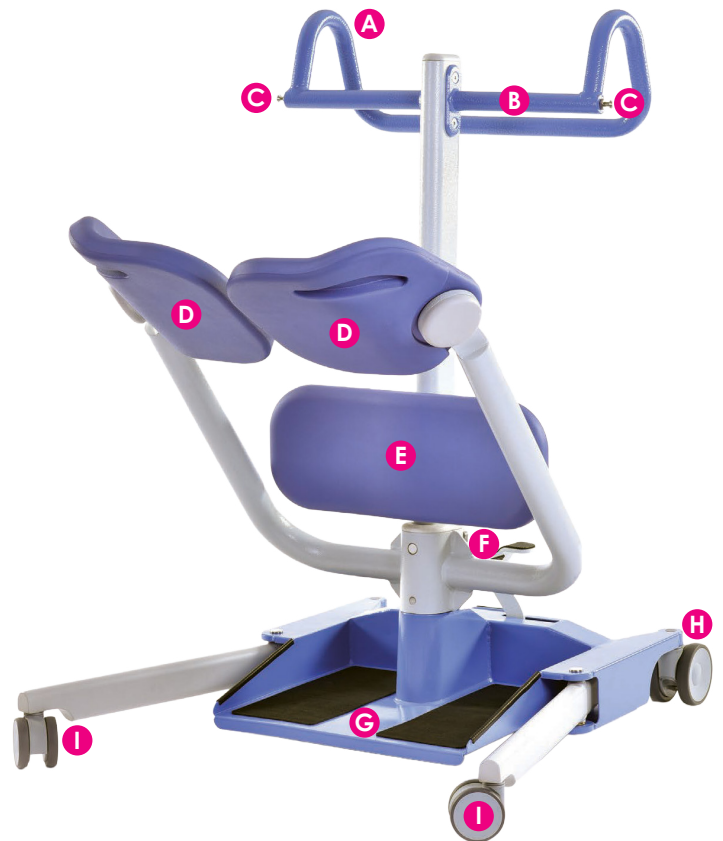

### Disassembles for Storage and Transportation

The Up can be quickly and easily disassembled into three components to ease storage and transportation requirements. Its compact footprint when disassembled enables it to fit comfortably into the boot of most family saloons.

Reassembly is straightforward and can be completed in under one minute, without the need for tools.



## Adjustable Leg Opening

The legs can be opened and closed to help navigate around bulky furniture such as commode frames and armchairs. This enables closer and safer access for the patient and better ensures good posture and positioning during use.



## Contoured Swing-away Seat Pads

Padded and shaped for comfort, each seat pad rotates outwards to allow unobstructed access for the patient when standing.

## Ergonomic Design Considerations

The Up has been designed with the **five** key principles of moving and handling in mind. This direct correlation ensures Oxford's are intuitive, safe, and comfortable to use.

- 1 Get close to the load
- 2 Use a wide stable base
- 3 Ensure a comfortable firm grasp
- 4 Keep spine close to neutral
- 5 Make sure movement is smooth



Soft moulded knee pad for comfort



Foot 'push pad' helps reduce the force needed to initiate forward movement.




Contoured, multi-point push handle and patient hand-hold bar



## Specifications

Product Code	TM0600
Height <b>A</b>	1093mm
Length <b>B</b>	960mm
Width <b>C</b>	633mm
Legs Open: External Width <b>D</b>	840mm
Legs Open: Internal Width	720mm
Legs Closed: External Width <b>E</b>	585mm
Legs Closed: Internal Width	470mm
Leg Height	105mm
Knee Pad Height   Footplate to Top of Pad	480mm
Foot Plate Height	37mm
Ground Clearance	17mm
Turning Radius	1100mm
Front Castors	75mm
Rear Castors   Braked	100mm
Safe Working Load (SWL)	200kg
Intended Operating Environment	+5°C - 40°C
Compliance	ARTG 397217
Warranty	12 months
<b>Weights</b>	
Base	20kg
Seat Support Tubes	9.5kg
Mast	4.5kg
Total	34kg

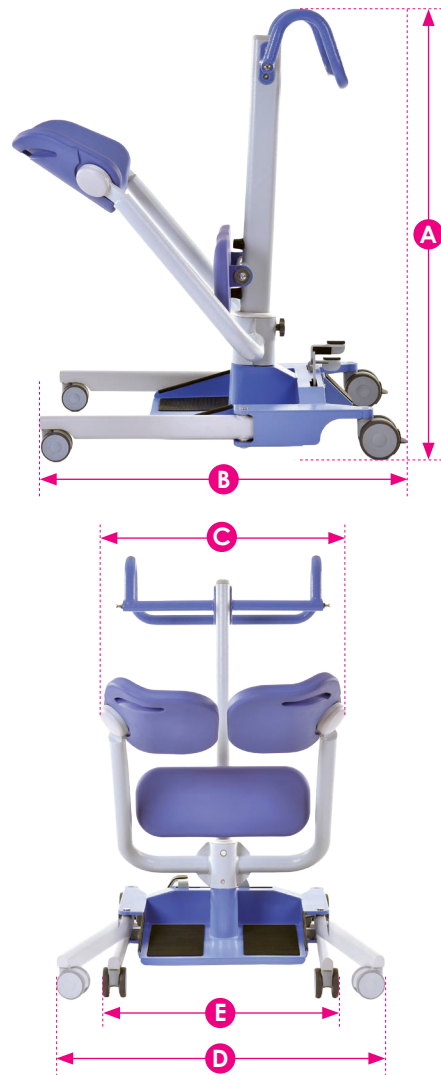

## Testing Standards

The Oxford® Up is fully compliant with the following standards and directives:

- ISO 10535
- BS EN 12182

## Oxford Deluxe Standing Sling with Clips

Can be used to provide additional seated support and reassurance during transfer.





# rehabhire & sales

1 300 000 030

320 Lorimer Street Port Melbourne  
contact@rehabhire.com.au  
rehabhire.com.au

# rehabhire & sales **REPAIRS AND SERVICE**

REGULAR MAINTENANCE OR REPAIRS

1 300 000 030

Unit 6 | 52 Wirraway Drive Port Melbourne  
service@rehabhire.com.au  
rehabhire.com.au/repairs-and-service

# rehabinstallation

CAN MODIFY EVERY ROOM IN YOUR HOME TO  
ENSURE IT IS ACCESSIBLE, SAFE, AND COMFORTABLE

1 300 000 030

320 Lorimer Street Port Melbourne  
contact@rehabinstallation.com.au  
rehabinstallation.com.au



Registered NDIS Provider