

BARDWELL

SAFETY MATTING

ABN: 54 482 168 988

HOIST MAT

INSTALLATION & CLEANING GUIDE



- **Roll out your mat into position beside the bed.**
- **Peel off the wax paper on the underside of your mat at each end.**
- **Then simply stick your mat to the floor and step on it to secure.**
- **The adhesive tape can be easily pulled up at any time.**
- **However, once you have positioned your mat, we don't recommend that you pick up or move your mat frequently as the adhesive backing will lose its tackiness.**
- **Your mat can be easily cleaned with a mop or a vacuum.**

rehab**hire** & **sales**

310 Lorimer Street Port Melbourne Victoria 3207

t: 03 9646 7200 | e: contact@rehabhire.com.au | www.rehabhire.com.au