

IC30 ACTIVEX™ MATTRESS




Pressure Redistribution Information

IC30 MATTRESS

Risk Level: High
SWL Minimum 20kg
SWL Maximum 228kg

Waterlow ≤ 20
Immersion Rating: Medium
Envelopment Score: 4/5


Subject 1: Female. 58kg
Average Peak Pressure: 14.3mmHg
Support area: 2379cm²

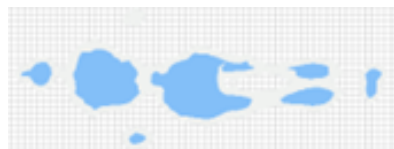

Subject 2: Male. 84kg
Average Peak Pressure: 18.7mmHg
Support area: 2548cm²



Scale mmHg

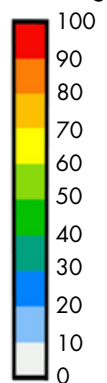

Risk category ratings, immersion ratings and envelopment scores are based on average peak pressure, therapist recommendations, case studies and Waterlow scale. For some high risk clients, Icare Zerotec covers are recommended. The best way to assess a client on a pressure-reducing surface is to monitor pressure points with visual checks over a period of five days.