


# Durable, Lightweight and **Portable**




The new **i-Go** from Pride Mobility, features advanced folding technology, enabling it to be quickly transported with ease. The i-Go features front suspension, durable seating system, under seat storage and much more. Designed to fit inside any small space, the i-Go is the perfect choice for the active individual.

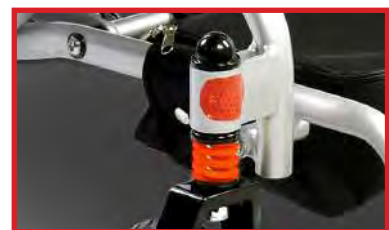

Hi-Torque front suspension



# Specifications PWS654175

Weight Capacity	120 kg (264.5 lbs.)
Maximum Speed <sup>2</sup>	Up to 5.95 km/h (3.7 mph)
Ground Clearance <sup>1</sup>	4.4 cm (1.73")
Overall Length <sup>1</sup>	87.63 cm (34.5")
Overall Width <sup>1</sup>	60 cm (23.6")
Turning Radius <sup>1</sup>	90 cm (35.4")
Standard Controls	Micon Control System
Total Weight (Without batteries)	19.8 kgs (43.6 lbs.)
Battery Weight (each) <sup>3</sup>	1.35 kgs (2.9 lbs.)
Suspension	Front
Range Per Charge	20 km
Battery Requirements	2 x 6.6 Ah, Li-ION Battery
Seating	Saddle back seat
Tyres	<b>Front castors:</b> 17.08 cm solid <b>Rear castors:</b> 21.06 cm solid
Colour	Silver 

## Notes:

1. Due to manufacturing tolerances and continual product improvement, this specification can be subject to a variance of + or - 3 %.
2. Specification varies with user weight, terrain, battery type, battery condition and tyre pressure.
3. Battery weight may vary based on manufacturer.

**Note: All specifications subject to change without notice.**

The information contained herein is correct at the time of publication; we reserve the right to alter specifications without prior notice. Speed varies with user weight, terrain type, battery charge, battery condition and tyre pressure.

## Features

- Large secure storage compartment
- Unique comfort seating
- HI-Torque front suspension
- Micon Control System
- Under seat storage

## How To Fold



- 1 Switch off and remove joystick



- 2 Place joystick in backrest pocket and remove cushion



- 3 Fold backrest down, then pull both arm rests anticlock-wise toward the back



- 4 Pull the two black toggles (located on the back) together



- 5 Push down on the seat frame until unit fully flattens



- 6 The i-Go in now ready to go...



AFY125.1 231019



**PRIDE**  
MOBILITY  
LIVE YOUR BEST®