### JAY® J3 Back

#### Important Consumer Information

**NOTE:** This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

**SUPPLIER:** This manual must be given to the user of this product.

**USER:** Before using this product, read this entire manual and save for future reference.

**WARRANTY REGISTRATION:** To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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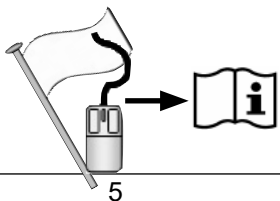
Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a J3 Back is appropriate.

### **⚠ WARNING!**

Backs should only be installed by:

- an authorized Sunrise Medical supplier.
- a qualified and competent healthcare professional who has undergone appropriate certified training by either sunrise medical or their approved supplier.

## **THE JAY J3 BACK INTENDED USE**

The JAY J3 Back is intended to provide a back support for a wide variety of wheelchair users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

The back can be installed onto a wide range of wheelchairs.

Maximum user weight:

300 lbs. (136 kg) – compact (and old standard hardware)

500 lbs. (227 kg) – heavy duty compact hardware

The intended lifetime of this product is 5 years.

### **⚠ WARNING**

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

### **⚠ WARNING**

Do not use J3 Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

### **⚠ WARNING**

Do not use the J3 back handle to lift wheelchair. It is intended only for back removal.

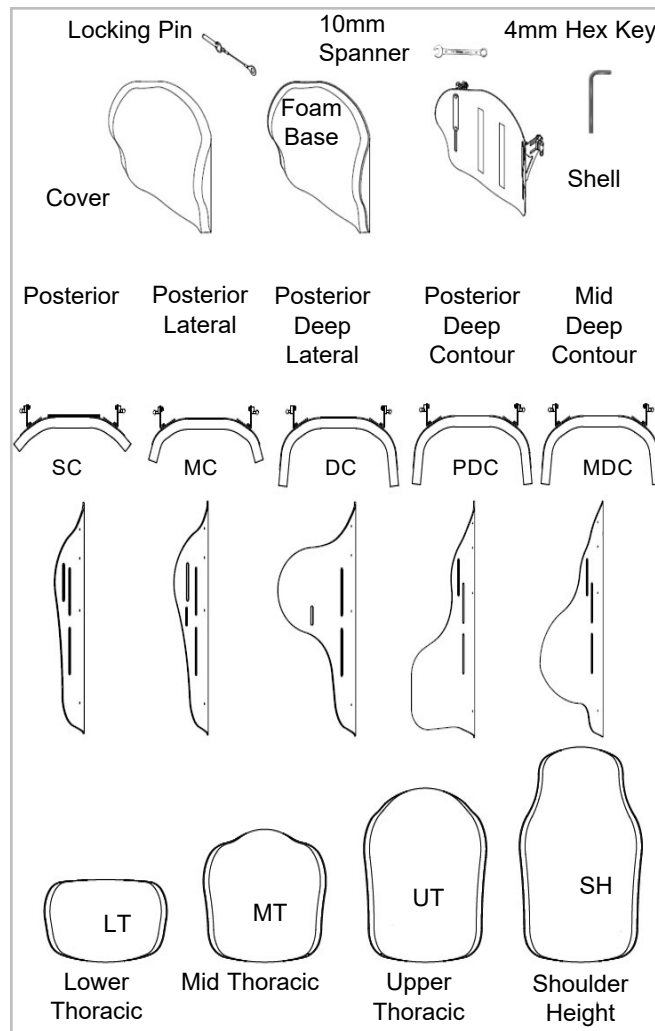
### **⚠ WARNING**

Carelessness causes fire.

### **⚠ WARNING**

Please read this manual before usage.

## **J3 BACK**

## MAINTENANCE

Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be retightened according to the installation instructions. All fasteners should be tightened to torque specifications as stated in the hardware installation section. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

## EASY CLEANING

**NOTE:** When disassembling the back for cleaning, note the position of the Velcro® Spine Align Components for correct reassembly.

### To clean back cover

Remove cover by detaching Velcro. Detach cover from back shell. Locate zipper on bottom of the cover and unzip. Remove foam and turn cover inside out. Zip cover closed to avoid snagging. Machine wash in warm water (60°C). Drip or tumble dry on low heat.

### NOTE:

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.

Reinsert the foam into the cover. Please note, the base has two layers of foam. When reinserting the foam, ensure that the thinner “light grey” foam is facing the shell. Once the foam is secured inside the cover, zip cover closed.

### To clean back shell

Shell may be wiped clean with soap and water. No other maintenance should be required.

### To clean positioning components

Positioning components can be wiped clean with soap and water. Do not immerse positioning components underwater.

If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

### Hygiene measures when being re-used

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray. To do this, you must use a disinfectant as authorised/recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly. Please be aware of the manufacturer's instructions for the disinfectant you are using. In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

### Conditions for safe storage

- Products should be stored in a dry place: at 20 – 75% Relative Humidity.
- Protect from direct sunlight and dust for long periods.
- Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
- Position and stack the packaged products according to the “this side up” arrows on the packaging.
- If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
- Do not stack anything on top of the packaged products when the “do not stack” mark on the packaging is shown.
- Product should never be stored outside and subject to the elements.

| Laundry Care Symbols                                                                |                                       |
|-------------------------------------------------------------------------------------|---------------------------------------|
|  | Machine wash in warm water (60° C)    |
|  | Only non-chlorine bleach, when needed |
|  | Drip dry                              |
|  | Do not iron                           |
|  | Do not dry clean                      |

## DETERMINING COMPATIBILITY

### Wheelchair Types

The J3 Back is designed to be compatible with most wheelchairs with the following exceptions:

Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

### **⚠ WARNING**

**Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.**


### Wheelchair Dimensions

The J3 Back is designed to replace the wheelchair's sling upholstery. Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in Figure 1. Then using that measurement, check Table 1 to determine if the back will fit your chair.

Next check that the back canes are a compatible diameter.

The J3 back hardware can be mounted on wheelchair back canes ranging from 3/4" (1.9 cm) to 1 1/8" (2.85 cm). Measure your back cane diameter (Fig. 2) and refer to Table 2 to ensure your back canes are compatible.

If you cannot determine that the back canes are compatible please contact your authorised Sunrise Medical provider or Sunrise Medical customer service.

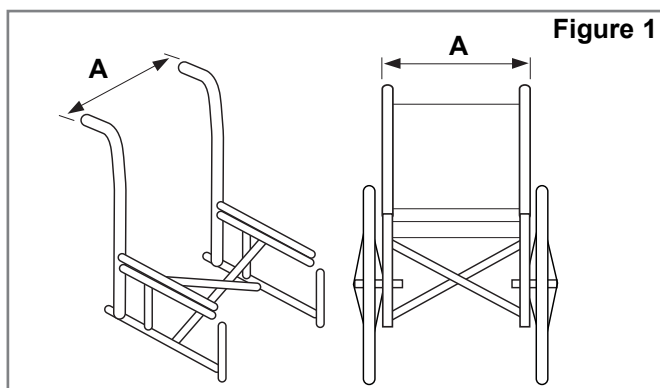
**Figure 1**

**Table 1**

| J3 Back Width*<br>(all contours) |      | Width of back canes from outside to outside of the tubes, (Fig.1-A) |      |               |      |
|----------------------------------|------|---------------------------------------------------------------------|------|---------------|------|
|                                  |      | Minimum Width                                                       |      | Maximum Width |      |
|                                  |      | inch                                                                | cm   | inch          | cm   |
| 12"                              | 31cm | 11.7                                                                | 29.7 | 14.3          | 36.3 |
| 14"                              | 36cm | 13.7                                                                | 34.9 | 16.3          | 41.3 |
| 16"                              | 41cm | 15.7                                                                | 40.0 | 18.3          | 46.4 |
| 18"                              | 46cm | 17.7                                                                | 45.1 | 20.3          | 51.6 |
| 20"                              | 51cm | 19.7                                                                | 50.1 | 22.3          | 56.5 |

\* Back Width is identified in your back's part number.

Example: J3SCMTS36 is a 36 cm back.

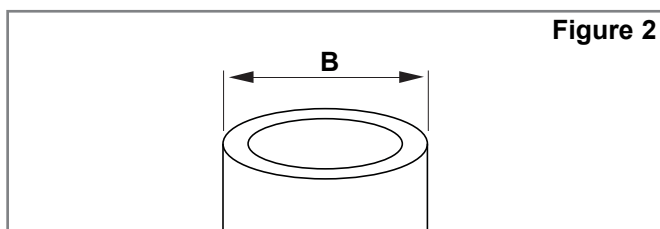**Figure 2**

**Table 2**

| J3 Back Width<br>(all contours) | Cane diameter (Fig. 2-B) |     |               |      |
|---------------------------------|--------------------------|-----|---------------|------|
|                                 | Minimum width            |     | Maximum width |      |
|                                 | inch                     | cm  | inch          | cm   |
| 14-20" (36-51cm)                | 3/4                      | 1.9 | 1-1/8         | 2.85 |

## INSTALLING THE MOUNTING HARDWARE

Please read the following instructions before beginning installation. To install the J3 Back, it is best to begin without a user in the wheelchair. Once the back has been installed and minor adjustments are needed the user can be seated back in the wheelchair.

### Tools required (included with J3 Back)\*

- 4mm hex key

- 10mm box end wrench

**NOTE:** Wherever torque settings are specified, it is strongly recommended that a torque meter (not included with JAY J3 Back) is used to verify correct torque specification is achieved.

### Hardware Installation Instructions (Figs. 3, 4, 5)

Valid for all hardware types (compact, heavy duty compact and old standard).

1. Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.
2. Visually determine the desired location of the receivers on the wheelchair's back canes. The mounting hardware should be at approximately equal heights on each back post (Fig. 3) and parallel to the seat frame (Fig. 4). Higher locations on the back canes are generally better.
3. Attach the hardware receivers. (Fig. 5)
  - a. Loosen the clamp screws (A) with the enclosed 4mm hex key. NOTE— For large tube diameters, clamp screws may have to be removed completely.
  - b. Attach and align receiver to the wheelchair back canes (B).
  - c. Hand tighten clamp screws (A). NOTE— Receivers may require additional lateral adjustment to ensure proper alignment.

### Attaching and Aligning the J3 Back Shell (Fig. 6)

- a. Loosen bracket nuts (C) and mounting pin nuts (F) using 10mm box-end wrench until hardware can move easily in all directions.
- b. Attach the J3 Back to the hardware receivers by inserting left and right mounting pins (D) in the left and right receivers (E).
- c. Locate back to the approximate location, relative to both chair and user.
- d. Hand tighten nuts on the inside of the mounting pin (F). NOTE— Back depth and angle may require additional adjustment to ensure proper fit to the user.
- e. Hand tighten bracket nuts (C). NOTE— Back height may require additional adjustment to ensure proper fit to the user.
- f. Test back release by pressing the release levers forward and removing the back off the chair. (See Removal and Replacement section for further details.) Proper alignment of the mounting hardware and back is achieved when smooth attachment and release can be easily done.
- g. If proper alignment has not been achieved, adjust the components until properly aligned.
- h. Once properly aligned, tighten the hardware receiver clamp screws. The recommended torque specification is 75 - 85 in-lbs (8.5 - 9.3 Nm).

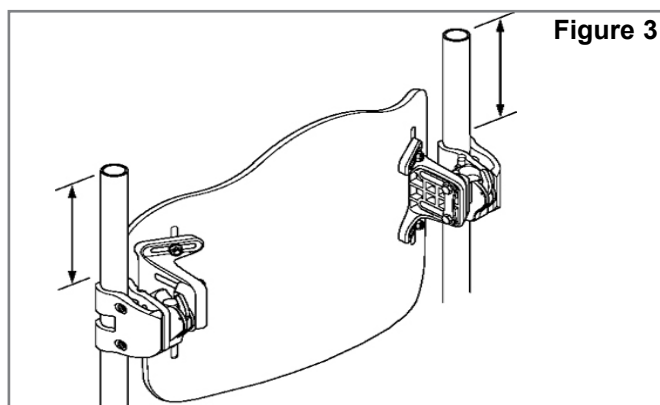
Figure 3

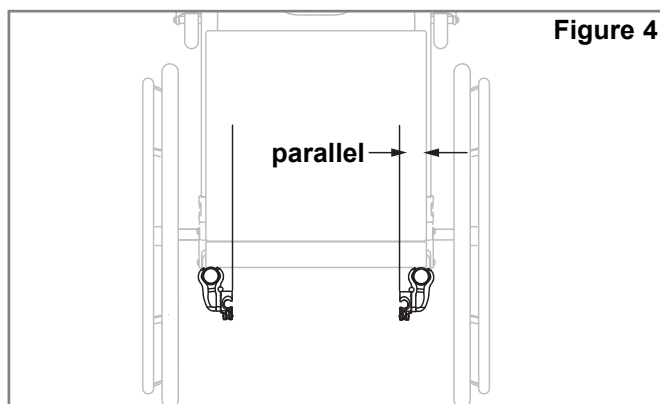
Figure 4

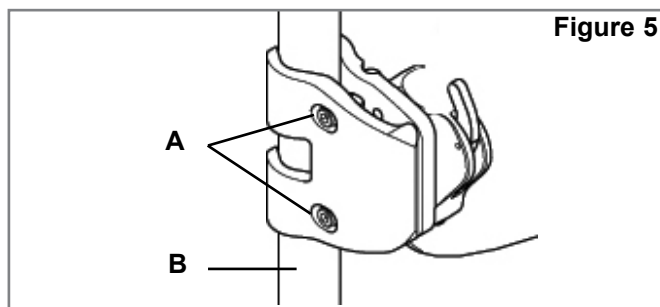
Figure 5

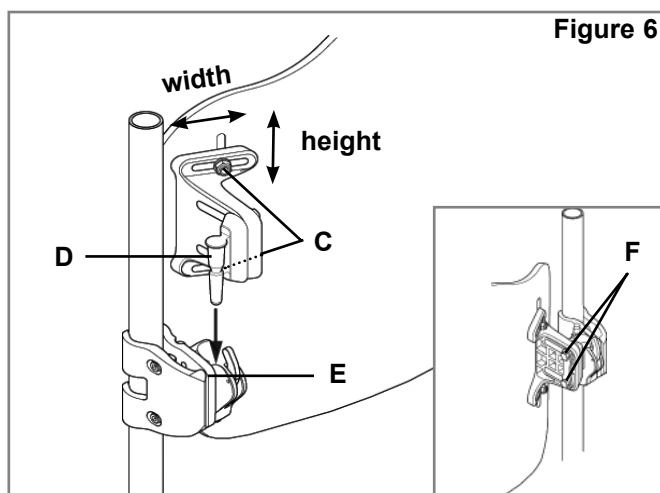
Figure 6

## ADJUSTING J3 HEIGHT, DEPTH AND ANGLE

The first phase of the J3 Back hardware installation is complete. It is now the appropriate time to have the user seated back in the chair for personalizing the J3 Back. Do not have the user apply full pressure on the back as it is not fully tightened to the full torque requirements.

### 1. Height Adjustment (Fig. 10)

- With the user in the chair, identify the desired height of the back. Once identified, have the user remove pressure from the J3 Back by leaning or sliding partially forward if possible to allow the adjustment to take place.
- Loosen the bracket nuts (A) and adjust the height of the back shell to required position.
- Securely tighten the bracket nuts (A). The recommended torque specification is 75-85 in-lbs. (8.5 - 9.3 Nm).

**NOTE:** After height adjustment is complete, check that the width was not inadvertently changed.

### 2. Depth/Angle Adjustment (Figs. 11, 12 & 13)

Please note there are two types of hardware that can be ordered— standard hardware (Fig. 11) or the longer, adjustable hardware (Fig. 12). Both sets of hardware are adjusted the same, but the standard hardware allows for 2 inches (5 cm) of depth adjustment (Fig. 11) while the extended hardware (adjustable hardware) allows for a total of 3.4 inches (8,65 cm) of depth adjustability (Fig. 12).

- To adjust J3 Back depth and angle loosen left and right nuts (Fig. 13-B) on bracket slots.
- Adjust depth and/or angle and securely tighten. The recommended torque specification is 100-110 in-lbs. (11.3 - 12.4 Nm). Ensure that none of the depth adjustment nuts (Fig 13-B) will slip or rotate.
- If additional depth is needed, remove back from receivers. Remove screws (Figs. 11-D, 12-D) from receiver using the enclosed 4mm hex key, and reposition to desired depth. Note: Prior to replacing the screws the application of a suitable medium strength threadlocker e.g. loctite 243 is strongly recommended to the receiver and screw threads, if the back is to be repositioned. Tighten screws to 25-35 in-lbs. (2.8 - 4 Nm).

### WARNING

Ensure that all hardware has been properly tightened once all adjustments have been completed and the J3 Back is ready for use.

### WARNING

To ensure optimal security of the back fitment, the locking pin must be inserted before the backrest can be used, (Fig. 14 & 15).


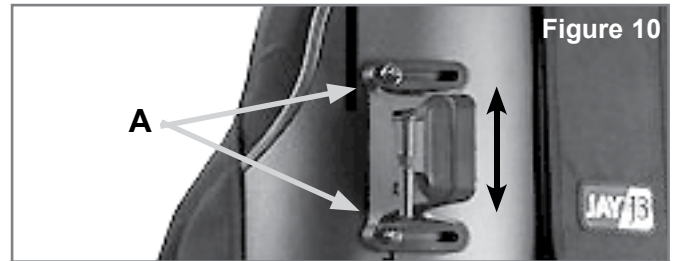
Figure 10

### Compact Hardware— 2 depth settings

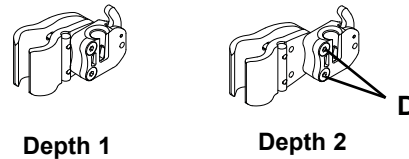Figure 11


Depth 1

Depth 2

### HD Compact Hardware

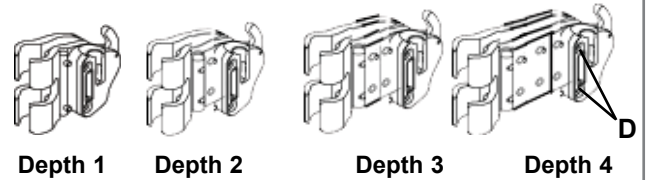Figure 12

Depth 1

Depth 2

Depth 3

Depth 4

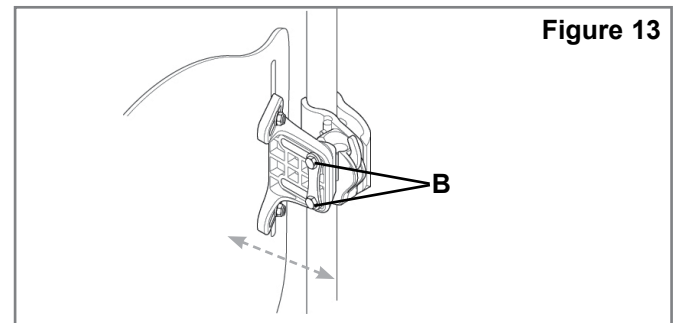
Figure 13

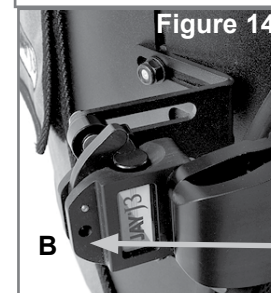
Figure 14

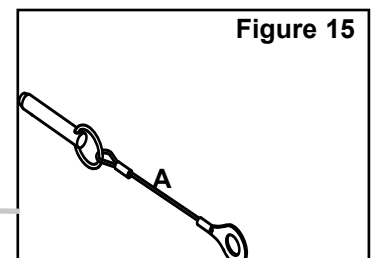
Figure 15

## VANITY FLAP INSTALLATION

1. Lift upper flap of J3 Back Cover (Fig. 17)
2. Match hook to loop and attach Vanity Flap to the back of the J3 Back Cover (Fig. 17). Reposition upper flap of cover to sandwich the Vanity Flap between the flaps the upper and lower flaps (Fig. 18).
3. Remove Seat Cushion from wheelchair and sandwich "tail" of Vanity Flap between wheelchair support surface and Seat Cushion (Fig 20).
4. Adjust Vanity Flap as necessary by moving the tail in or out from under the Seat Cushion. If necessary the upper part of the vanity flap can be moved up or down on the Seat Back also.
5. Insert Hook & Loop Wrap ties through the appropriate grommets on the Vanity Flap, then wrap around the wheelchair frame and insert the leading end through the slot end of the Wrap (Fig. 21). Snug up and close outside gaps in Vanity Flap (Fig. 22).

### Optional Storage Bag

**NOTE:** There is an optional storage bag available to attach onto the vanity flap, (Fig.16).

Figure 17

Figure 18

Figure 19

Figure 20

Figure 21

Figure 22

Figure 16

Figure 23

Finished J3 Vanity Flap Installation

## SPINE ALIGN POSITIONING COMPONENTS

Spine Align Positioning Components are designed to provide intimately shaped support within an “off-the-shelf” back. (Depending on your market, Spine Align components may be included with your back.) To install Spine Align please read the following directions.

### 1. Spine Align Positioning Component Installation

- Have the user remove pressure from the J3 Back by leaning or sliding partially forward if possible.
- Release Velcro strip on the top cover flap (Fig. 24) and lift the cover forward to reveal the inside of the J3 Back.
- Place the positioning component(s) on the Velcro inside of the shell (Fig. 25) and reattach the top cover flap. Pull down making certain that the top seam is aligned with the top of the shell. If seam does not line up, the bottom of the cover may need to be loosened (Fig. 26).
- Have the user test the current Spine Align setup for comfort and support and repeat steps a-c until adjustment is complete.
- Multiple configurations and combinations of Spine Align components can be used. If additional Spine Align shapes or sizes are required, please contact your dealer or Sunrise Medical.

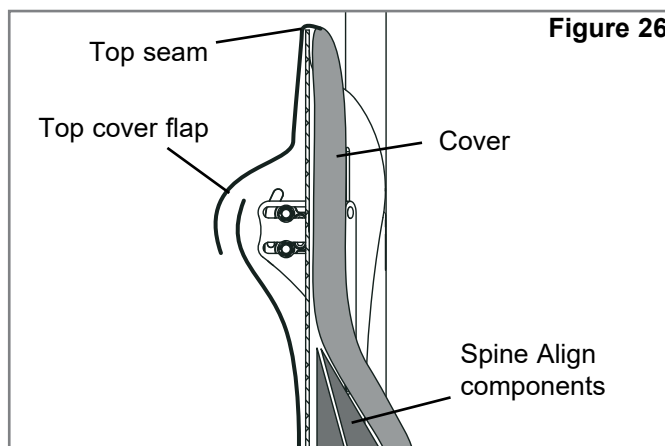
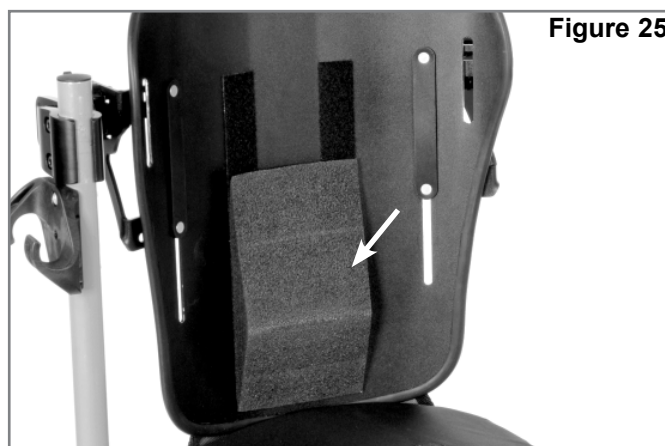
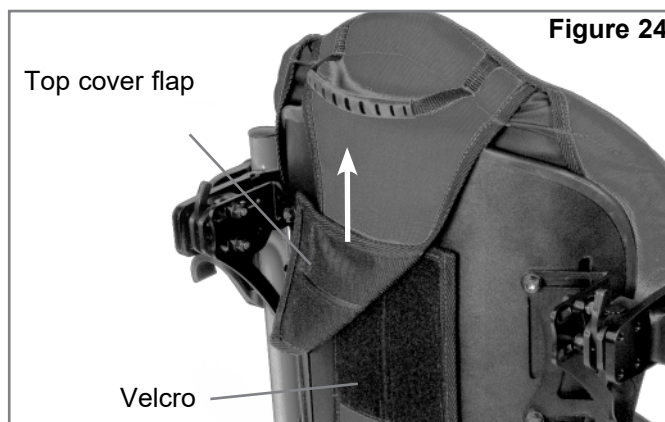
**NOTE:** If you require further information on the J3 Spine Align system, please review our animation at [www.sunrisemedical.com](http://www.sunrisemedical.com).

Installation of the J3 back is complete.

## J3 ACCESSORIES

If you have ordered J3 accessories such as laterals, headrests or harnesses please consult the installation instructions related to those items.

| Positioning Components   |                                                                                     |
|--------------------------|-------------------------------------------------------------------------------------|
| Large Lumbar             |  |
| Small Lumbar             |  |
| Wedge                    |  |
| Curved Shim              |  |
| Flat Shim                |  |
| Posterior/Lateral Pelvic |  |
| Lateral Shim             |  |
| Large Bean Bag           |  |
| Small Bean Bag           |  |

## BACK TRANSIT SAFETY

The J3 back has been dynamically tested for use in a motor vehicle. Please follow all installation, use and maintenance instructions within this manual as well as the transit instructions listed below.

## WARNING

**FAILURE TO HEED THESE WARNINGS COULD CAUSE SEVERE INJURY TO THE WHEELCHAIR RIDER OR OTHERS.**

## INSTRUCTIONS

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraint.
2. The distance between the top of the user's shoulder and the top of the back should not exceed 6.5" (16.5 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. The Wheelchair Tiedown and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
6. The wheelchair must be forward facing during transport.
7. In order to reduce the potential of injury to vehicle occupants wheelchair-mounted accessories such as trays and respiratory equipment should be removed and secured separately.
8. This product is intended for use by adults ranging in weight from 79 lbs (35 kg) to 300 lbs (136 kg).
9. Postural support devices such as pelvic positioning belts, anterior trunk supports or postural supports such as lateral trunk supports should not be relied on for occupant restraint in a moving vehicle unless they are labeled as conforming to ISO Standard 16840-4.
10. Alterations or substitutions should not be made to the structure, parts or components, without consulting the manufacturer.

## REMOVAL AND REPLACEMENT

### Removal

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. To remove, push each release lever forward to unlock (Fig. 27). Once both levers are in the forward position, simply lift the back straight up (Fig. 28) using the handle located on the top of the back.

### Proper reattachment after removal

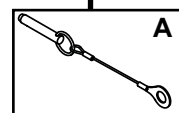
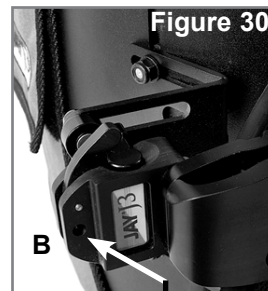
To reattach the back to the hardware receivers, simply line up the pins to the receivers and push into place (Fig. 29). If properly inserted the pins will “click” to a locked position.

### WARNING

**If you are uncertain if you have heard the locking “click”, pull up on the back using the handle to ensure that it has been securely locked.**

### Locking Pins (Fig. 30)

The J3 Back comes with two locking pins which, when secured, prevent the back from being released from the chair. To lock, insert the pins (A) into the mounting receivers (B). Simply remove the pins if the quick-release capability is desired.

**CONTRAINDICATIONS:**** WARNING**

Do not use or prescribe this backrest for anyone exceeding the stated Maximum User Weight.

**RECYCLING / RE-USE:**** WARNING**

- Risk of Injury
- Always have an authorized Sunrise Medical dealer inspect the product for damage or wear prior to transferring it to another user. If any type of damage is found, DO NOT USE THIS PRODUCT.
- Failure to heed these warnings may lead to severe injury.

**This Product is suitable for re-use\*:**

- Have an authorized Sunrise Medical dealer inspect the shell, mounting hardware, foam and cover for damage or wear. If any damage is found, do not re-use.
- Clean and disinfect the shell, mounting hardware foam and cover in accordance with local government regulations. Refer to your local Infection Control Office for advice.
- After cleaning and disinfection, have a authorised Sunrise Medical dealer re-inspect the shell, hardware, foam and cover to insure that the cleaning/disinfection process did not cause any damage.

** WARNINGS:**

- Failure to follow the re-use directions can lead to Severe Injury of the User.
- When being re-used, this product should be adapted and fit to the new user by a authorized Sunrise Medical dealer or clinician.

**24 MONTH LIMITED WARRANTY****24 MONTH LIMITED WARRANTY**

Each JAY J3 Back is carefully inspected and tested to provide peak performance. Every JAY J3 Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

|                                                                                             |                                    |                                                                                              |                                          |
|---------------------------------------------------------------------------------------------|------------------------------------|----------------------------------------------------------------------------------------------|------------------------------------------|
| <b>Type:</b>                                                                                | Product Name/<br>SKU Number        |  XXXX-XX-XX | Date of<br>Manufacture                   |
| <b>Date:</b>                                                                                | LOT: defined by<br>Julian Date     | <b>Part Code</b>                                                                             | Assembly Part<br>Number                  |
| <br>XXX mm | Seat width                         | <b>MD</b>                                                                                    | This symbol means<br>Medical Device      |
| <br>XXX mm | Seat depth                         |             | Manufacturer's<br>address                |
| <br>XXX kg | Max user weight                    | <b>EC REP</b>                                                                                | European<br>Authorised<br>Representative |
| <b>CE</b>                                                                                   | CE Mark                            | <b>UK RP</b>                                                                                 | UK Responsible<br>Person                 |
| <b>UK CA</b>                                                                                | UKCA Mark                          | <b>CH REP</b>                                                                                | Swiss<br>Representative's<br>address     |
|            | Consult<br>instructions for<br>use |             | Importer's address                       |

**SAMPLE**

**JAY**

Part Code:

Date:  

cm 

cm 

kg 

**Max**

Sunrise Medical (US) LLC  
2842 Business Park Avenue  
Fresno, California, 93727 USA

**EC REP**

Sunrise Medical GmbH  
Kahlbachring 2-4  
69254 Malsch / HD  
Germany

**UK RP**

Sunrise Medical Ltd.  
Thorns Road, Brierley Hill  
West Midlands DY5 2LD  
United Kingdom

Assembled in Mexico

106132 Rev. J

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product meets the requirements in accordance with EU and UK regulations. Options or accessories shown are available at extra cost.

The varieties of backrest fitting variants, as well as the different positioning components to deliver postural support, pressure distribution, increase seating tolerance are benefits according to each user requirement and indication, mean that it can be used by a range of users with different positioning needs and indications as follows:

- Spinal Cord Injury
- MS
- Neurological disease pattern (hemiplegia, M. Parkinson)
- Muscle Dystrophy
- Amputation
- Tetraplegia
- Joint contracture
- Cerebral Palsy
- Extremity defect deformity

No contraindications are known or available.

**NOTE:** General user advice. Not following these instructions may result in physical injury, damage to the product or damage to the environment!

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

#### B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used. Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

#### Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at [www.Sunrisemedical.co.uk](http://www.Sunrisemedical.co.uk). All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745. Guidance on the combination, such as mounting, can be found at [www.SunriseMedical.co.uk](http://www.SunriseMedical.co.uk).

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