

## Thermal Compression Gloves

8\*192 replaces 8\*199 gloves. (Sized Support)  
NEW simplified design for comfortable fit.

- Front Grip where it is needed.
  - Streamlined smooth back of hand to reduce friction.
  - Relief from arthritic pain, cold and sensitive hands
  - Textured and fingerless gloves for sensitivity
  - Breathable and moisture wicking
  - Clinically validated
- 
- Sizes are the same: XS, S, M, L, XL & XXL
  - Product pack stays the same: size 2
  - Product Dimensions: 125(H)x95(W)x40(D) weight 102, 104, 106, 108 & 110, 112gm

Materials:

- Nylon, Polyester, Spandex Rubber Foam



Actual Packaging



**THERMOSKIN®**  
to move is to live

# Thermal Compression Gloves

## INSTRUCTIONS FOR USE

- Shaped for comfort, the Thermal Compression Gloves provide warmth and even compression for the temporary relief of arthritic pain.
- Designed for everyday use, the textured palms and fingertip cut outs provide additional grip and a better sense of touch so you can get on with your day-to-day without restriction.

## WARNINGS

- Do not wear within the first 72 hours following an acute injury. This product contains a synthetic fibre, which may cause allergic reactions. Should an irritation or rash develop, discontinue use and seek medical advice. If pain persists, discontinue use and consult your medical professional.

## CARE INSTRUCTIONS

- Wash before wear. It is recommended to place the product in a delicates washing bag. Machine wash with cold water. Drip dry. Do not wring, tumble dry, bleach or dry in direct sunlight.

## WARRANTY DESCRIPTION

- Covered by a 12 month guarantee against faulty workmanship or materials

## MATERIAL

- Outer lining: Nylon Mid layer: Rubber foam Inner lining: Polyester (Does not contain latex) - Made in Thailand

## RELATED INJURIES

- Pain, stiffness or inflammation of the finger joints, Weakness in joints, Visible deformities in the fingers

## INSTRUCTIONS

- Position the product on the right and/or left hand so that the fastener strap is wrapped around the lower forearm. Secure the strap.

## OCCASION TO BE WORN

- Everyday, 72 Hours Post Injury

SIZING GUIDE

Measure  
circumference  
around knuckles

	SIZE	INCHES	CM
—	XS	6-6 3/4"	15-17
—	S	7-7 3/4"	18-20
—	M	8-9"	21-23
—	L	9 1/4-10 1/4"	24-26
—	XL	10 3/4-11 1/4"	27-29
—	XXL	11 3/4"+	30+



Actual Packaging

