

reabhire & sales

YOUR  
LEADING  
HEALTHCARE  
EQUIPMENT  
PARTNER



# XXL Rollator King

## User Manual

Strong and functional rollator designed for the bariatric user.



1300 000 030 | [reabhire.com.au](http://reabhire.com.au)

YOUR  
LEADING  
HEALTHCARE  
EQUIPMENT  
PARTNER

# Table of content

|                                   |    |
|-----------------------------------|----|
| 1. Safety .....                   | 3  |
| 2. Symbols .....                  | 4  |
| 3. General .....                  | 5  |
| 4. Using the Rollator King .....  | 5  |
| 5. Assembling the rollator .....  | 6  |
| 6. Adjustment .....               | 6  |
| 7. Rollator Support .....         | 7  |
| 8. Cleaning and maintenance ..... | 8  |
| 9. Disposal .....                 | 8  |
| 10. Warranty .....                | 9  |
| 11. Specifications .....          | 10 |
| 12. Ordering .....                | 11 |
| 13. Accessories .....             | 11 |

# 1. Safety


---



## CAUTION

Do not use the Rollator King without prior thorough instruction by trained staff or by persons who have received instructions in the use of the rollator. Improper operation may cause injury to the user.

If the Rollator King is used for persons weighing more than the maximum user weight, the risk of damaging the Rollator King increases.

Carefully study these instructions before using the Rollator King. Pay attention to Safety Instructions indicated by a warning triangle .

Cobi Rehab accepts no responsibility for damage caused to the product nor for personal injury caused by improper installation, assembly or use of the Rollator King.

Contact Cobi Rehab immediately in the case of damaged or missing parts. Never use your own spare parts.



## SAFETY INSTRUCTIONS

Always lock the rollator before sitting down.

Always remember to tighten the hand nuts for height adjustment.

Never sit on the rollator and push off with the feet.

Never use the rollator on stairs and escalators.

Never use the rollator as a wheelchair.




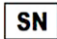





Never use the rollator to transport large items. Only items that fit in the basket.

Never use the rollator on surfaces that slope by more than 3,5 degrees.

Never use the rollator on soft and uneven surfaces.

Keep both feet on the ground while using the rollator as resting chair. This prevents the rollator from tipping backwards if the user leans against the backrest.

## 2. Symbols

| The following symbols are used in this user manual and/or on the product label.        |   |
|--|---|
|       | Caution!  |
|       | Declaration of conformity, Medical Device Regulation, EU 2017/745 |
| IP31   | Ingress Protection Ratings Guide.                                 |
|       | Cobi Catalogue number   |
|       | Serial number   |
|       | Date of manufacture   |
|       | Manufacturer  |
|       | Medical device  |
|      | Body weight   |
|     | Consult instructions for use                                      |
| CE-Label on the product is located on the bottom frame.<br>See example of label below. |   |



XXL-Rehab Rollator King  
0172-050-000



325 KG

MADE IN TAIWAN



2000204



2020

(01)05740000102490(21)2000204

## 3. General

---

This user guide contains important information about the use of Rollator King and the target group for the rollator. It also includes a wide range of specifications on Rollator King.

The most recent version of this user guide can be found on our website. Check if the version you are reading now is the most recent.

If you need a user guide that is written in a larger font size, the guide is available in PDF format at [cobirehab.com](https://www.cobirehab.com), which can be scaled up to the required font size.

You can always contact Cobi Rehab on + 45 7025 2522 or [cobi@cobi.dk](mailto:cobi@cobi.dk) if you want more information about Rollator King.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

The UDI number of the rollator appears on the label, which is located on the side of the frame.

Product safety notices and product recalls can be viewed on the website.

<https://www.cobirehab.com/kunde-hos-cobi/vigtig-produktinformation/>

## 4. Using the Rollator King

---

Rollator King is intended for users weighing up to 325 kg. The rollator is designed for use outside and inside, in institutions, care centers and private homes.

⚠ The rollator should only be used on level ground and accessible terrain, indoors as well as outdoors.

⚠ Be aware that the user always walks with the rollator close to the body. Too long distance can result in instability.

### 4.1 More use

Rollator King is available in two sizes, 610 mm and 710 mm which refers to the distance between the handles. Both models have a seat width of 510 mm, seat height of 510 mm and a seat depth of 370 mm.

The total width of the Rollators is 130 mm more than the distance between the handles. Be aware of the model with a width of 840 mm can have difficulties going through certain doorways.

#### How to choose the right size

Which size of Rollator King depends on measures and the body shape of the user. Contact Cobi

Always consult a therapist before assigning a Rollator King to a person.

#### Contraindications

The user of Rollator King must have physical and cognitive resources to control and operate the rollator in a safe manner.

Rehab for guidance and instructions on how to choose the right size.

For users with a lot of fullness around the pelvis and thighs, we recommend a size 710 mm.

Always measure user standing. For example, place a book or similar on the side of the user's body where the user is widest. Then measure the distance between the two books.

## 5. Assembling the rollator

---

In addition to the rollator, the shipped box also contains two handles (connected to the rollator by cables), a basket and a safety backrest.

1. Unfold the rollator.
2. Remove the screws for height adjustment from the handlebar.
3. Attach the bars to the rollator.
4. Mount the back on the front of the rollator by pulling upwards on the knobs on the left and right sides, respectively.

Notice that the knob can be locked to make it easier to mount the back. This is done by pulling the knob upwards and turning it a quarter turn. Remember to loosen the knob again to fasten the safety back rest.

5. Mount the basket in the front by placing the two hooks on the upper tube below the seat. Place the notch on the bottom of the basket on top of the bottom tube.

6. Adjust the height of the handles to suit the user.
7. Adjust the brakes.

⚠ Do not use the rollator without the safety backrest.

The safety backrest ensure that the weight is not positioned too far forward. If the weight is positioned too far forward, the rollator may tip.

⚠ Do not lay or put extra cushions on the upholstered seat when in use.

## 6. Adjustment

---

### Adjusting the height of the handle

Remove the screw and the height-adjustment knob from both handles to enable the handles to move freely up and down.

2. Push the handle up or down until the required height has been found. Reinsert the screw and the knob again. Place the bolt on the back side, so it fits in the hole.

3. Tighten.

Ensure left and right handles are level to avoid instability problems.

⚠ Make sure the user keeps with the Rollator King close to the body when in use. The correct size of the rollator is there for important to make sure the user can place between the handles.

### How to fold the rollator?

1. Grab the front of the seat (the side closest to user) with one hand. Hold one of the handles with the other hand.

2. Lift the seat.

3. The rollator now folds.

To minimize the size of an already folded rollator, remove the back and basket.

### Adjustment of the brakes

The rollator can brake in two ways:

Brief stops:

Squeeze each brake handles upwards.

Longer stops:

Push both brake-handles downwards, so they lock in a "down" position. The handles stay in this position until they are actively released.

### Brake inspection

The brakes are properly set on delivery.

Brake operation, and parking brake should be checked before every use.

If the parking brake cannot lock or if the brakes are not working correctly, contact Cobi Rehab immediately.

## 6.1 Transportation in motor vehicle

The Rollator King can be transported from one place to another. Fold the rollator as described above.

⚠ Rollator King cannot be used as a seat in a motor vehicle.

Rollator King withstands being lifted in the frame, handles or back rest, but not in the basket.

Contact Cobi Rehab for more information on transportation or disassembly.

## 6.2 Storage (incl. long-term storage)

The Rollator King should be stored in the dark and covered with a sheet or other cover.

The Rollator King should be stored at temperatures between -10° and 50°C.

Check regularly that the rollator has not become wet or damaged during storage.

# 7. Rollator Support

---

Rollator Support is in the construction like Rollator King. Only difference is the height adjustable forearm supports which will assist the user to drive the rollator forwards in an upright position and at the same time relieve the wrists.

If a user needs forearm support for a short or longer time they can be mounted on the frame of Rollator King.

Contact Cobi Rehab for more information on mounting of forearm supports.

⚠ Please note that forearm supports are only possible if the user weighs under 200 kg.

See "Accessories"- section for Rollator Support item numbers.



## 8. Cleaning and maintenance

---

### 8.1 Cleaning and disinfection

The frame, seat handles, and safety backrest is cleaned with water and a household detergent.

Contact Cobi Rehab for further guidelines on cleaning and maintenance.

#### Disinfection

The frame, seat handles, and safety backrest can be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm.

⚠ The upholstered seat and the back rest must be replaced or thoroughly disinfected when Rollator King is transferred to a new user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

The seat and back holder can be replaced when switching to a new user.

### 8.2 Maintenance

Check bolts and nuts every three months. Retighten them if needed.

⚠ Non-compliance with the bolt inspection guidelines can lead to malfunction.

## 9. Disposal

---

Products that can no longer be used are to be disposed of in a sustainable way. Please contact the relevant authorities for information on proper disposal.

#### Recycling

Parts of the rollator can be reused. Contact Cobi Rehab for more information.

# 10. Warranty

---

When delivered, the Rehab Rollator King is free from defects. Should, contrary to all inspections, there be any defects or damage to the Rollator King, please contact Cobi Rehab immediately.

The Rollator King is covered by a 1-year warranty from the date of purchase. The invoice serves as warranty certificate.

The warranty does not apply to:

- products whose serial number, batch number etc. have been removed or considerably damaged
- products that have been repaired by unauthorized staff
- normal wear and tear, for example cleaning, adjustment, lubrication and update of equipment or parts.

## 10.1 Unintended incident

Report any incidents related to the use of Rollator King to Cobi Rehab.

## 10.2 After expiry of expected service life

After the expected service life, it is important that an overall assessment of the product is made before continued use.


After the expected service life, Cobi Rehab cannot guarantee the suitability and safety of the product, as Cobi Rehab has no control over how the product is used and wear and tear.

The warranty is void if Cobi Rehab assesses that the Rollator King has been used for purposes other than its intended use, for example, if its weight capacity is exceeded.

In case of incorrect use, both service and spare parts will be invoiced.

The warranty described above is the only valid warranty.

Cobi Rehab may not be held liable for damages of any kind.

 Lack of maintenance can damage the product, shorten the life, or affect the performance of the Rollator King.

The overall assessment of the product must be carried out by qualified professionals, and as a minimum, consideration must be given to how the product has been used, the condition of the product and its components.

Cobi Rehab can always make such an assessment.

# 11. Specifications

| <b>Model 61 cm and 71 cm</b>  |   |
|---|---|
| Maximum user weight   | 325 kg  |
| Total length  | 850 mm  |
| Total width   | 740/840 mm                                    |
| Width between handles   | 610/710 mm                                    |
| Height adjustable in 2.5 cm intervals   | 770.5 - 990 mm                                |
| Seat width  | 510 mm  |
| Seat depth  | 370 mm  |
| Seat opening  | 610/710 mm                                    |
| Seat height   | 510 mm  |
| Weight  | 15.7 kg                                       |
| Weight capacity in basket   | 5 kg  |
| Foldable  | Yes   |
| Massive wheels  | Yes   |
| Frame   | Steel/SAE4130                                 |
| Frame and seat  | Flame retardant                               |
| Seat and back   | Polyurethane                                  |
| Wheels  | 8" massive                                    |
| Expected product life   | 7 years                                       |
| ISO-classification  | ISO 12 06 06 - 11                             |
| Risk class  | I   |
| UDI-DI  | 5740000102490 (61 cm) / 5740000102506 (71 cm) |
| Basic UDI   | 5740000100007N8                               |
| The surface temperature of the Rollator King can increase or decrease when exposed to external sources of heat or cold. |   |

## 12. Ordering

---

| <b>Model</b>                   | <b>Seat opening</b> | <b>Total width</b> | <b>Item number</b> |
|--------------------------------|---------------------|--------------------|--------------------|
| Rollator King 61, incl. basket | 610 mm              | 740 mm             | 0172-050-000       |
| Rollator King 71, incl. basket | 710 mm              | 840 mm             | 0172-071-000       |

## 13. Accessories

---

| <b>Accessories</b>        | <b>Item number</b> |
|---------------------------|--------------------|
| Rollator Support Kit/King | 0170-999-010       |



# rehabhire & sales

1 300 000 030

320 Lorimer Street Port Melbourne  
contact@rehabhire.com.au  
rehabhire.com.au

# rehabhire & sales **REPAIRS AND SERVICE**

REGULAR MAINTENANCE OR REPAIRS

1 300 000 030

Unit 6 | 52 Wirraway Drive Port Melbourne  
service@rehabhire.com.au  
rehabhire.com.au/repairs-and-service

# rehabinstallation

CAN MODIFY EVERY ROOM IN YOUR HOME TO  
ENSURE IT IS ACCESSIBLE, SAFE, AND COMFORTABLE

1 300 000 030

320 Lorimer Street Port Melbourne  
contact@rehabinstallation.com.au  
rehabinstallation.com.au



Registered NDIS Provider