

rehabhire & sales

YOUR
LEADING
HEALTHCARE
EQUIPMENT
PARTNER



XXL Chair

User Manual

For use by: Bariatric users at home, at care centers, or in waiting rooms.



1300 000 030 | rehabhire.com.au

YOUR
LEADING
HEALTHCARE
EQUIPMENT
PARTNER

Instructions for XXL-Rehab Patient/Dinner chair

CAUTION



The XXL-Rehab Patient/Dinner chair must not be used without proper instruction by a qualified professional.

Using the XXL-Rehab Patient/Dinner chair for individuals whose weight exceeds the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage to the product or personal injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do not use substitute parts.


Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff made without written authorisation from Cobi Rehab.


 To ensure proper use of the XXL-Rehab Patient/Dinner chair, please read these instructions thoroughly before using the Patient/Dinner chair. In doing so, pay particular attention to the warnings and safety instructions marked with .

XXL-Rehab Patient/Dinner chair may only be assembled and adjusted by qualified staff who has been instructed in the proper use of the Patient/Dinner chair. Incorrect assembling and adjustment can injure the user.

Use

The XXL-Rehab Patient/Dinner chair is designed for bariatric users weighing up to 325 kg. The chair is used at hospitals, in waiting rooms or at home at the dinner table.


 If used by someone weighing more than 325 kg, there is a risk that the XXL-Rehab Patient/Dinner chair will break or sustain damage.

 To avoid tipping, always use the XXL-Rehab Patient/Dinner chair on level, hard flooring with a maximum slope of 3°.

Before use

The bolts for adjusting the legs and backrest must be properly mounted and tightened on the side of the XXL-Rehab Patient/Dinner chair.

During use

 Pay attention to unsafe lifts or movements during transfer from and to the XXL-Rehab Patient/Dinner chair.

The seat back can be adjusted in three different positions. Adjusting the depth setting on the seat back does not affect the stability of the XXL-Rehab Patient/Dinner chair.

The surface temperature of the XXL-Rehab Patient/Dinner chair can increase or decrease when exposed to external sources of heat or cold.

Maintenance and cleaning

The frame components and the seat and armrests may be cleaned with water and ordinary detergent. To disinfect the seat use alcohol. Make sure to fasten the seat properly again, if removed for disinfection.

Regularly check all bolts and nuts, and retighten if necessary.

⚠ Failure to inspect the bolts may cause malfunction.

Warranty

When delivered, the XXL-Rehab Patient/Dinner chair is free from defects. Should, contrary to all inspections, there be any defects or damage to the Patient/Dinner chair, please contact Cobi Rehab immediately.

The XXL-Rehab Patient/Dinner chair is covered by a 1-year warranty from the date of purchase. The invoice serves as warranty certificate.

The warranty does not apply to:

- products whose serial number, batch number etc. have been removed or considerably damaged
- products that have been repaired by unauthorised staff
- normal wear and tear, for example cleaning, adjustment, lubrication and update of equipment or parts.

The warranty is void if Cobi Rehab assesses that XXL-Rehab Patient/Dinner chair has been used for purposes other than its intended use, for example, if its weight capacity is exceeded.

In case of incorrect use, both service and spare parts will be invoiced.

The warranty described above is the only valid warranty.

Cobi Rehab may not be held liable for damages of any kind.

Specifications

Maximum weight capacity is	325 kg
Total length	80-101 cm
Total outer width on armrests	74 / 84 cm
Total height	79-94 cm
Seat width	61 / 71 cm
Seat depth	40-52 cm
Seat height	44-59 cm
Height of backrest	37 cm
Distance from floor to seat underside	33-48 cm
Distance between armrests	65 / 75 cm
Seat to armrest	28 cm
Weight	16.6 / 17.6 kg
Frame	Powder-lacquered chromium molybdenum steel
Seat / back / armrests	Polyurethane
ISO - classification	ISO 09 33 03
Expected product service life	7 years

Disposal

Products that can no longer be used are to be disposed of in a sustainable way. Please contact the relevant authorities for information on proper disposal.



XXL-Rehab Patient/Dinner chair is manufactured and tested in accordance with the international DS/EN 12182:2000 standard.



rehabhire & sales

1 300 000 030

320 Lorimer Street Port Melbourne
contact@rehabhire.com.au
rehabhire.com.au

rehabhire & sales **REPAIRS AND SERVICE**

REGULAR MAINTENANCE OR REPAIRS

1 300 000 030

Unit 6 | 52 Wirraway Drive Port Melbourne
service@rehabhire.com.au
rehabhire.com.au/repairs-and-service

rehabinstallation

CAN MODIFY EVERY ROOM IN YOUR HOME TO
ENSURE IT IS ACCESSIBLE, SAFE, AND COMFORTABLE

1 300 000 030

320 Lorimer Street Port Melbourne
contact@rehabinstallation.com.au
rehabinstallation.com.au



Registered NDIS Provider