

# Mobil-2


## User Manual

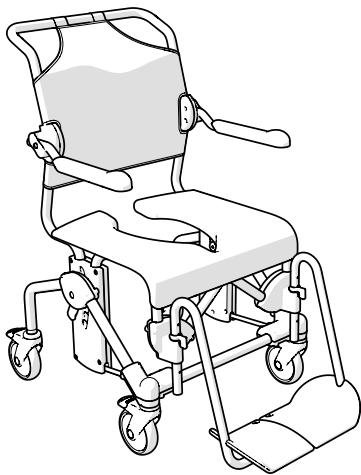Swift Mobil-2  
Swift Mobil-2 XL

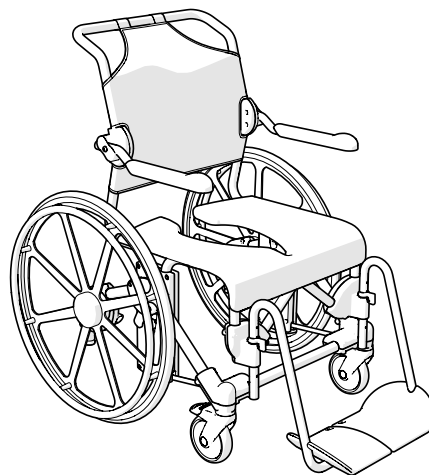Swift Mobil 24''-2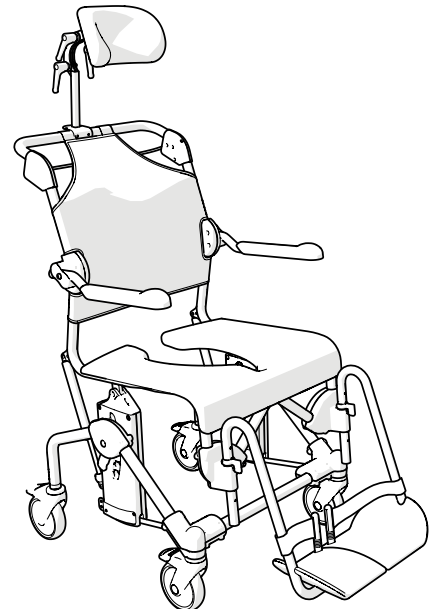

Swift Mobil Tilt-2  
Swift Mobil Tilt-2 XL

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Swift Mobil-2	Swift Mobil-2 XL	Swift Mobil 24"-2	Swift Mobil Tilt-2	Swift Mobil Tilt-2 XL
●	●	●	●	●
			●	●
●	●	●	●	●
	●			
●	●	●	●	●
●	●	●	●	●

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## General

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at [www.etac.com](http://www.etac.com). You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

Item list: ● = Applies to, ○ = Accessories

See the item list on the right which displays the information applicable to each product.

The images which are referred to on each page are found at the start of the manual (pages 3 to 12). See Figures A to F for the corresponding text.

### Device description

The Swift Mobil-2 family is a range of mobile shower commode chairs whereof Swift Mobil-2 and Swift Mobil Tilt-2 are assistant propelled and Swift Mobil 24"-2 can be propelled by the user. The devices can be adapted to specific user's needs through several adjustments and accessories.

### Intended purpose

Swift Mobil-2, Swift Mobil-2 XL, Swift Mobil 24"-2, Swift Mobil Tilt-2 and Swift Mobil Tilt-2 XL (hereafter also referred to as "the device" or "the product"), are medical devices intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to be used when performing hygiene tasks in the shower, at a sink, or over the toilet, as well as for transfers to and from the hygiene room.

### Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. They are intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more.

Secondary users of the device are caregivers providing assistance and clinicians/technicians who set up the device.

### Intended environment

The device is intended for indoor use at home environments or institutions and is suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

### Intended application

The device is intended for short-term and long-term use and can be applied several times per day.

The device is intended for use with intact skin. The device is intended for refurbishment and reuse.

### Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see [www.etac.com](http://www.etac.com).

### Indications

Indication for use is disability, including, but not limited to, an individual with physical disabilities who needs backrest and/or armrests to maintain a sitting position and assistance with transfers to/from the chair.

### Contra indications:

There are no known contraindications.

### Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

Make sure the castors are locked when the user sits down or gets up from the chair. Risk of injury.

A defective device must not be used. Risk of injury.

Check user sensitiveness for pressure ulcer before use

A heated device can cause burns

Body parts may get pinched while applying or removing the pan.

Ignoring the instructions for use may result in injury.

### Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device is tested and meets the requirements of the EN ISO 10993-1 and ISO 17966:2016.

### Pre-purchase information

Pre-purchase information can be found at [www.etac.com](http://www.etac.com).

### Adverse event

In case of an adverse event occurred in relation to use of the device, it must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

### Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

## en Device description

See figure A

**A1.** Bolt **A2.** Leg support **A3.** Seat **A4.** Arm support upholstery **A5.** Back support **A6.** Item no label (ser.no. manufacturing date) **A7.** Product label

**A8.** Rear wheel **A9.** User brake

**A10.** Head support

## Symbols, labels

See figure B

### Symbols in the manual and on the device:

**B1.** Warning, precaution or limitation. **B2.** Useful advise and tips. **B3.** Material for recycling. **B4.** The part/function is locked and can not be moved/adjusted. **B5.** The part/function is unlocked and can be moved/adjusted. **B6.** Click on the icon or scan the QR code for video. **B7.** Showing option **1** of two possible options. **B8.** Showing option **2** of two possible options. **B9.** Tool Torx. **B10.** Tool Allen key. **B11.** Max user weight (see technical data). **B12.** Read instructions for use. **B13-B14.** Cleaning (see service and maintenance). **B15.** Locked wheels. **B16.** Do not stand on the foot support, risk of tipping. **B17.** Seating position affects the stability.

### Information on item no label:

**B18.** Product name. **B19.** Product description. **B20.** Serial number. **B21.** Item number. **B22.** CE marked **B23.** Medical device **B24.** Production date. **B25.** Bar code acc. to GS1-128 GTIN-14 and serial no.\*

\*The device's date of manufacture can be read from the barcode on the device. The number **11** is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

## Technical data

See figure C

## Handling

See figure D

### Installation upon delivery

#### Transfer over a threshold:

Chair only, push down on the wheel attachment.

With occupant, pull the chair backwards.

#### In stationary position:

Point the castors forward and activate the brakes to reduce the risk of tipping when the chair is used stationary and when transferring. Risk of injury if the device tips over.

D8

Do not sit on the front edge of the seat. Risk of injury if the device tips over.

Always lock as many castors as possible when the chair is to be used in a stationary position.

#### Transferring:

Point the castors forward and activate the brakes to reduce the risk of tipping when the chair is used stationary and when transferring. Risk of injury if the device tips over.

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as risky. Visit [www.etac.com](http://www.etac.com) for manual transfer aids.

D1-D5

Make sure the castors are locked when the user sits down or gets up from the chair. Risk of injury.

D6

Fold up the armsupport on the transfer side of the chair.

D7

Carry out the transfer.

D8-D14

#### Arm supports:

The arm supports are foldable.

D13

#### Foot supports:

The assistant can easily raise and swivel the leg supports to the side for the user to sit down in or get up out of the chair. The foot supports are detachable.

D14

Do not stand on the foot plates. Risk of injury if the device tips over.

D9

#### Tilt function:

The assistant can adjust the sitting angle with ease by holding in the handle and tilting to the required position. Make sure that the wheels are locked if the tilt function is used when the user is sitting in the chair.

D15

## Adjustment

See figure E

### Height adjustment:

The height is easily adjusted by inserting the bolt in the required position. See distances in the technical data section. Check that both sides have been adjusted to the same height.

**E1-E7**

Do not use the chair if the height adjustment bolt is unlocked. Risk of injury.

### Arm supports:

The height of the arm supports and/or width between the arm supports can be adjusted. Loosen the screws, twist the support to the desired position and retighten the screws. Then loosen and refit the arm support plate in the correct position. Swift Mobil Tilt-2 XL is delivered with the arm supports mounted in the most inward position (seat width 48 cm).

**E8**

Fit the gas piston in position A to tilt the seat at angles from -5° to 30°

**E9**

Fit the gas piston in position B to tilt the seat at angles from 0° to 35°

**E10**

### Foot support:

Various height adjustment options are available by moving the locking pin. The leg supports have 6 different height settings. If the user's feet are at risk of slipping backwards, heel straps are recommended.

**E11**

### User brake

**E12**

## Service and maintenance

Clean the device using a standard non-abrasive cleaning agent (pH between 5 and 9). Rinse and dry.

Disinfect the device with 70% disinfectant solution.

The device requires no other maintenance.

The device and its accessories can be safely cleaned at 85°C for 3 minutes (unless otherwise stated). This applies to reconditioning.

Constituent materials are resistant to common disinfectants.

The following surface treatment methods have been used to protect against corrosion:

Lacquered surfaces = Polyester powder coating

Non-lacquered steel surfaces = Fe/Zn-Ni

5-year guarantee against material and manufacturing defects. For terms and conditions, see [www.etac.com](http://www.etac.com).

## Transport, storage and disposal

### Storage

The device should be stored indoors in a dry place at a temperature above 5°C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

To be recycled according to national regulations.

Swift Mobil-2	Swift Mobil-2 XL	Swift Mobil 24"-2	Swift Mobil Tilt-2	Swift Mobil Tilt-2 XL
			●	●
●	●	●	●	●
●	●	●	●	●
	●			
	●			
●	●	●	●	●
●	●	●	●	●

Accessories	See figure F					
		Swift Mobil-2	Swift Mobil-2 XL	Swift Mobil 24"-2	Swift Mobil Tilt-2	Swift Mobil Tilt-2 XL
F1. Leg and calf support, angle-variable		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F2. Foot support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F3. Angle adj. foot support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F4. Amputee leg support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F5. Commode pan		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F6. Pan-holder attachments		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F7. Attachments for low pan-holder mounting		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F8. Commode pan holders		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F9. Bucket holders		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F10. Pan		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F11. Lid with handle		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F12. Splash guard		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F13. Soft Comfort seat, 2 cm high, open at back		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F14. Soft Comfort seat, 4 cm high, open at back		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F15. Comfort seat		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F16. Soft Comfort seat		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F17. Soft Comfort seat - narrow opening		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F18. Soft cushion - standard opening		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F19. Soft cushion - oval opening		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F20. Seat cover		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F21. Soft back pad		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F22. Comfort padding		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F23. Arm support cushion, pair		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F24. Trunk support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F25. Chest and hip positioning belts		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

Accessories	See figure F					
		Swift Mobil-2	Swift Mobil-2 XL	Swift Mobil 24"-2	Swift Mobil Tilt-2	Swift Mobil Tilt-2 XL
F26. Chest belt		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F27. Positioning belt		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F28. Crossbar 605 mm		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F28. Crossbar 665 mm		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F29. Anti-tip stabilisers including bracket		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F30. Anti-tip stabilisers excluding bracket		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F31. Tilter		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F32. Rear wheel kit		<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F33. Head support and bracket		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
F34. Heel straps		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
F35. Hemiplegic arm support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F36. Arm support lock		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F37. Extended back support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F38. Short leg support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F39. Angled leg support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F40. Widening kit for leg support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F41. Anti tip front		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F42. Back Swift Mobil-2, upright		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F43. Head support upright		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
F44. Cover leg- and calf support		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

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