

Splash guard

Model 2.322


Assembly instructions

Note:

The following assembly is to be carried out in a specialist workshop.


Application

The splash guard is designed for the assembly to drive wheels (1) and steering wheels (2) of model 2.322. On wet surfaces it prevents the clothes from getting dirty.

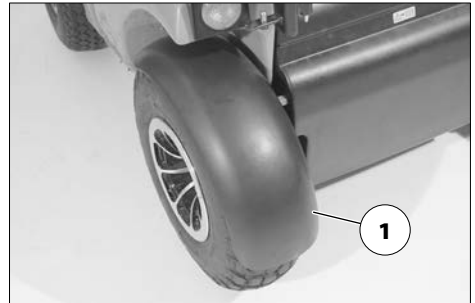

Overview

- (1) 1x Splash guard – front right
 - 1x Splash guard – front left
- (2) 2x Splash guard – rear
 - (a) 2x Distance bush, WW13 M6 x14
 - (b) 4x Flat head screw M6 x 12
 - (c) 4x Flat head screw M6 x 20
 - (d) 6x Washer B6.4 DIN 9021


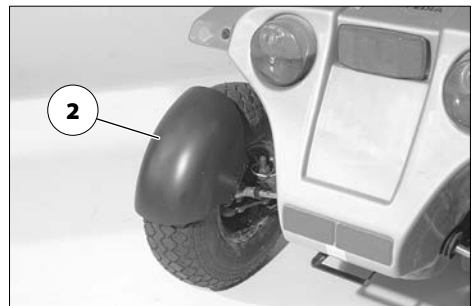


Safety information

- ⚠ Watch for jamming areas during the complete assembly. – Danger of crushing!
- ⚠ Jack the vehicle up securely during assembly (lifting jig)!
- ⚠ Observe the corresponding chapters of the respective operating manual!

Mounting the splash guard to drive wheels

1. Dismantle the drive wheels (1).
 2. Unscrew the screws (2) of the front revetment.
- ⚠ Assembly steps 3. to 5. do not apply when a hole already exists in the front revetment.
3. Mount the splash guard and front revetment to the rocker (3) with the new attachment screws (c).
 4. Drill the holes for the attachment of the distancers (a) into the front revetment.
 - ⚠ For this use the existing holes in the splash guard as a drilling template (4).
 - ⚠ Drill with a \varnothing 6.5 mm drill, at a right angle to the splash guard, max. 5 mm deep.
 5. Remove the splash guard and the front revetment.
 - ⚠ For this disassemble the attachment screws (c).


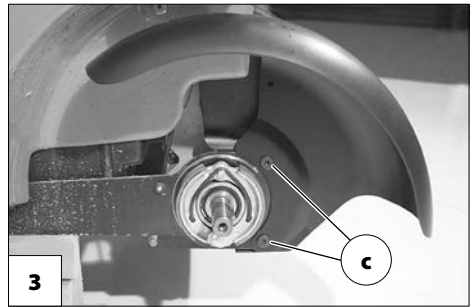
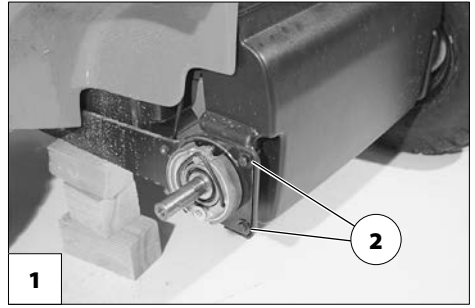


6. Mount the distancer (a) with the screws (b, d) to the front revetment on each side.

🔧 Fig. 1b shows the assembly for model 2.322 (from 2. quarter 2007).

🔧 Fig. 1b shows the assembly for model 2.322 (up to 1. quarter 2007).

7. Mount the splash guard and front revetment to the rocker (2) with the new attachment screws (c).
8. Screw the splash guard with the attachment screw (b) to the distancer (a) (2).
9. Remount the drive wheels (3).

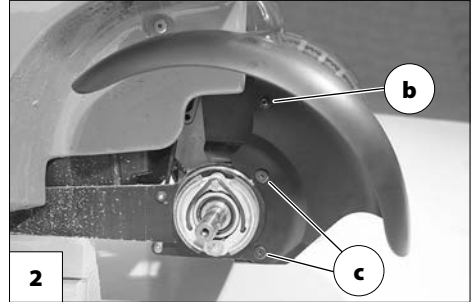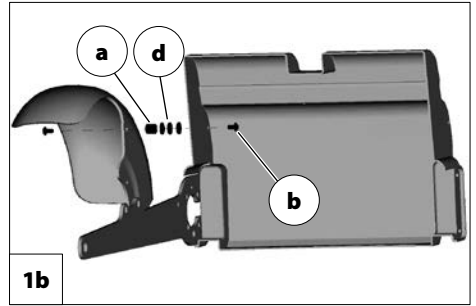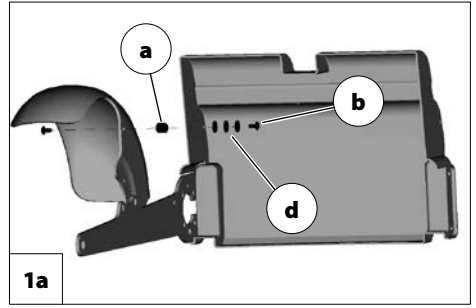

Mounting the splash guard to steering wheels

1. Dismantle the tie-rod (1).
 - 🔧 Counter the ball head with a WW14 mm combination wrench.
2. Place the splash guard onto the attachment plate and slide it to the centre of the wheel (2).
3. Mount the tie-rod with splash guard (3).
 - 🔧 Counter the ball head with a WW14 mm combination wrench.

Subsequent operations

Conduct a driving test and check for grinding or clattering noises.

Check the correct fit of the assembled parts.



MEYRA GmbH

Meyra-Ring 2
32689 Kalletal Kalldorf
GERMANY



Tel +49 5733 922 - 311
Fax +49 5733 922 - 9311



info@meyra.de

www.meyra.de



This product in combination with the electric wheelchair is conform with the EC-directive 93/42/EEC (MDD) and from 25th May 2021 fulfils the valid requirements of the regulation (EU) 2017/745 for medical devices.